



# Enuresis (Bed-Wetting) Treatment Systems

## Replacement Components/Parts


The DRI Excel® and DRI Eclipse® enuresis treatment alarms don't have any replaceable parts other than the urine sensors, called the Urosensor™, which are a major component of the alarm's success. What appears to be a very simple part of the product is in fact a complex design challenge. The sensor has a tough job – every night it's wet by urine and then washed over and over again; it gets tugged and pulled during sleep ("corded model") and it has the corrosive factor of urine to contend with.

**IMPORTANT NOTE: The sensor is the key to beating bed-wetting. Don't be fooled by tiny clips or domes on the underwear, the urine can miss (especially with boys) and not trigger the alarm!**

### Urosensor™ for DRI Excel® (Plug-In or "Corded" model)

Replacement remote sensor for DRI Excel® (#4600). Over the past **25 years** the creators have designed and evaluated the urine sensor again and again. The result is the Urosensor™: a **unique**, patented, **non-metallic**, completely molded, **flexible product**. It cannot corrode, irritate the skin or allow urine to leak inside and break. Because the urine cannot leak inside, it can be washed quickly to be instantly reused; it does not get contaminated with the urine, and internal wiring to the alarm cannot become corroded.

The Urosensor™ on the DRI Excel® (plug-in or "corded" model) is a consumable item and it is expected that it will need replacing from time to time depending upon its duration of use, just as earphones on a portable radio will need replacing from time to time. Remember that earphones may last varying times depending on how carefully or otherwise they are used, especially when being used by a child and so the life of the Urosensor™ will vary. It cannot be predicted exactly how long the sensor will last—for this reason our strong advice is to purchase a spare Urosensor™ when you buy your DRI Excel® so that there is no break in treatment while waiting for a replacement to arrive.

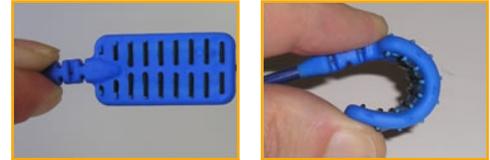

Color may vary

LEARN MORE



Sold by the each, 1 sensor per package.  
This is a stocked item.  
Non-Sterile, Latex-Free.

**LATEX-FREE**

Item No.	Description	Dimensions†	Case Item No.
<b>4603</b>	Urosensor™ (plug-in type)	Sensor Pad: 0.25" x 1.63" (0.64cm x 2cm x 4.3cm) Cord: 35" (89cm)	Not Available

### Urosensor™ for DRI Eclipse® ("Wireless" model)

Replacement remote sensor for DRI Eclipse® (#4610). The "wireless" Urosensor™ Remote presents an even greater challenge than the plug-in model. To be wireless, it has to communicate with the alarm by a radio signal. Therefore it has electronics and a battery inside. These have to be miniature to keep the sensor small and comfortable. It must be completely sealed to keep the urine out and the battery small yet durable enough to last for a long time not just 7 days or 1 month as in some other alarms.

The Urosensor™ Remote is the **only 100% wireless sensor to have solved all of these complex issues** which enable the sensor to be small, completely sealed, located right in the crotch of the underwear where it is triggered immediately by the urine and for the battery to last for at least three (3) years.



Color may vary

LEARN MORE



Sold by the each, 1 sensor per package.  
This is a stocked item.  
Non-Sterile, Latex-Free.

**LATEX-FREE**

Item No.	Description	Dimensions†	Case Item No.
<b>4613</b>	Urosensor™ Remote "wireless"	0.38" x 1" x 2" (1cm x 2.5cm x 5cm)	Not Available

† Approximate dimensions in inches(") and centimeters(cm).

‡ Distance depends somewhat on the situation and number of obstacles. Actual distance may vary.