



Urocare Products, Inc. • 2735 Melbourne Avenue • Pomona, California 91767-1931 • U. S. A.
T: (909) 621-6013 • F: (909) 621-4436 • E: sales@urocare.com • I: www.urocare.com

Uro-Bond® V Brush-on Silicone Adhesive



DESCRIPTION

Uro-Bond® V is a pressure-sensitive, brush-on, silicone adhesive specifically formulated for unique bonding applications. Uro-Bond® V is suitable for adhering non-permanent materials to the body(skin). Features include: It's a non-hazardous material (no special handling or storage is required), non-flammable, virtually odor-free, retains its adhesive qualities in the presence of moisture or perspiration, adhesive properties are unaffected by normal temperature variations or time and it adheres to many surfaces such as skin, metals, glass, paper, fabric, plastics, other silicone and organic rubber products.

APPLICATIONS

Uro-Bond® V may be used to adhere dressings, prosthetics and metal or plastic devices to the body. Suggested uses include, but are not limited to: Securing male external urinary catheters, adhering colostomy, ileostomy and tracheostomy appliances, attaching electrodes, affixing surgical dressings and pads, adhering maxillo-facial prosthetic devices and toupees. Products made in part or otherwise incorporating Uro-Bond® V should be thoroughly tested before use to determine the suitability of the product's performance in a specific application.

SAFETY AND FIRST AID MEASURES

Do not use near fire, flame, heat, electric sparks or any other source of ignition. No smoking. Use only as directed. **Keep out of reach of children.** Eye irritant. In case of eye contact, flush eyes immediately with water, remove any contact lenses and continue to flush eyes with plenty of water for at least 15 minutes—contact physician. Do not ingest or permit adhesive to enter stoma or open wounds. Harmful if ingested—ingestion may result in gastric disturbances. In case of accidental ingestion, rinse mouth and obtain medical attention immediately. Do not inhale. In case of accidental inhalation, remove person to fresh air. Give artificial respiration if not breathing. Oxygen may be given by qualified personnel if breathing is difficult—obtain medical attention. If redness, swelling or rash appears at or near area of use, discontinue use and contact Physician. **Most important symptoms and effects, acute and delayed:** Respiratory tract irritation signs or symptoms may include coughing, sneezing, nasal discharge, headache, hoarseness and nose/throat pain. Contact with the skin during normal product use is not expected to result in irritation. Moderate eye irritation signs/symptoms may include redness, swelling, pain, tearing, and blurred/hazy vision. Gastrointestinal irritation signs/symptoms may include abdominal pain, stomach upset, nausea, vomiting and diarrhea.

STORAGE AND SHELF LIFE

Do not store near fire, flame, heat, electric sparks or any other source of ignition. When stored in its original unopened container at or below 72° F (22° C), Uro-Bond® V has a shelf life of Two (2) years from the date of manufacture. CAUTION: Heat sensitive. **Do not store above 80° F (26° C).** If product thickens during storage, Urocare recommends the use of Uro-Bond® Thinner ("501003) to return Uro-Bond® V to the desired consistency. PLEASE NOTE: Do not leave jar open for prolonged periods of time—close cap securely immediately after application.

INSTRUCTIONS FOR USE

Uro-Bond® V is applied using the brush attached to the underside of the cap. To obtain the maximum adhesion, surfaces must be thoroughly cleaned and dry prior to application. The adhesive should be applied liberally to both surfaces which are to be adhered—additional coats of adhesive may increase adhesion. Allow both surfaces to become tacky before adhering.

REMOVAL

To remove Uro-Bond® V from the body(skin), use Urocare® Adhesive Remover Pads (item #5600) or another medical tape/adhesive remover. Do not use Uro-Bond® Thinner as an adhesive remover; it is strictly intended for use as a thinning agent only! To remove Uro-Bond® V from surfaces other than the body (skin), use one of the following removers or solvents: isopropyl alcohol (rubbing alcohol), ethyl alcohol, toluene, xylene, or hexane—it should be noted that these solvents may cause irritation or defatting of the skin and are highly flammable. They may also deteriorate certain surfaces like fine finishes or some plastics, and should be used with caution. Thoroughly test the use of any remover/solvent to determine suitability of use. Liberally apply the appropriate remover to a cloth/gauze pad and wipe clean.

PACKAGING

Uro-Bond® V brush-on silicone adhesive is supplied in two sizes: Item #500503, 3 fl. oz. (88.7 mL) and item #5005015, 1.5 fl. oz. (44.4 mL) glass jars with a brush attached to underside of cap.

ORDERING INFORMATION

Please visit our website at www.urocare.com for access to all current product information and a list of distributors. If you need further assistance, call customer support at 1-800-423-4441 weekdays from 7:30 AM to 4:00 PM (PST). The information contained herein is based on information we believe to be accurate and reliable; however, Urocare® encourages you to thoroughly test any application and independently conclude the effectiveness and suitability of Uro-Bond® V before use. Suggestions for use should not be taken as inducements to infringe upon any patent.



PLEASE RECYCLE
Printed on
Recycled Paper