




FITZ®-ALL FABRIC LEG STRAP SET FOR URO-SAFE® VINYL URINARY LEG BAGS


MAY BE USED WITH MOST VINYL URINARY LEG BAGS

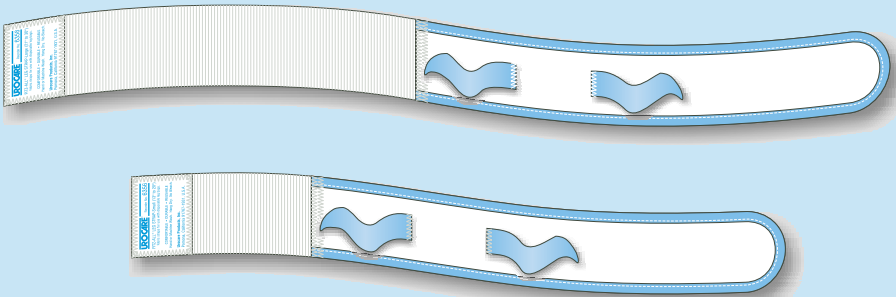

COMFORTABLE • WASHABLE • DURABLE & REUSABLE

ELIMINATE BRUISING, PRESSURE SORES,
PULLED HAIR, CIRCULATION RESTRICTION AND
BURNS CAUSED BY RUBBER STRAPS.

FULLY ADJUSTABLE SELF-GRIPPING FASTENERS
MAKE APPLICATION SAFE AND SIMPLE WITH A
GENTLE TOUCH.

PRODUCT NO.		DESCRIPTION & CONTENTS
	6370	Large—No. 6358 & No. 6356
	6360	Small—No. 6357 & Number 6355

NOTE: The elastic sections are made in part with natural latex rubber
which may cause an allergic reaction in some individuals.

READ THIS FIRST:

For best results, take certain precautions before using these straps. Be certain that there is no loose stitching or other irregularities before each use.

Those with known allergic reactions to certain materials (especially neoprene rubber) should not wear these straps. Discontinue use at first signs of dermatitis or any other type of skin irritation and contact a physician immediately.

Do not over-tighten straps; products made with elastic and/or neoprene rubber material can—if improperly fit—cause circulation restriction or skin breakdown; however, a bag strapped too loosely can slip down the leg and cause damage to male external or foley catheters by stretching them. Should any discomfort occur while wearing, discontinue use immediately and contact a fitter or physician.

To avoid possible drainage problems, read all instructions and precautions carefully before use. Only allow a urinary drainage bag to fill to its maximum capacity in emergency situations.

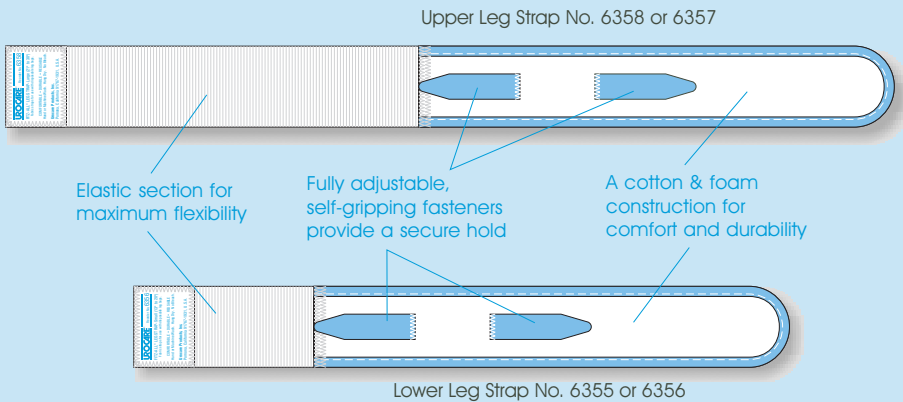


FIGURE 1

Available in 2 kits or sold separately in 4 different sizes.

PRODUCT NUMBER	DESCRIPTION & CONTENTS	DIMENSIONS ¹	
		INCHES (")	CENTIMETERS (cm)
FITZ-ALL® UNIVERSAL FABRIC LEG STRAP KITS FOR VINYL LEG BAGS			
6370	Large, contains straps No. 6358 & 6356	See below	See below
6360	Small, contains straps No. 6357 & 6355	See below	See below
FITZ-ALL® UNIVERSAL FABRIC LEG STRAP SINGLES FOR VINYL LEG BAGS			
6358	Large	21 through 30	53 through 76
6357	Medium	17 through 23	43 through 58
6356	Small	13 through 20	33 through 50
6355	Extra-Small	9 through 13	23 through 33

¹Approximate dimensions in inches and centimeters.

DIRECTIONS FOR USE:

1. Be certain that the bottom drain valve or thumb clamp of the Uro-Safe® leg bag is closed to prevent spills.
2. Thread the self-gripping fasteners through the strap eyelets in the four corners of the leg bag. Note: the fasteners must be threaded from under the leg bag and not over the inlet or outlet valve openings as shown in Figure 2.
3. Strap the leg bag to the inside thigh or calf of the desired leg, making certain that the inlet valve is in an upright position—see Figure 4. Then adjust the straps, but not too tightly.

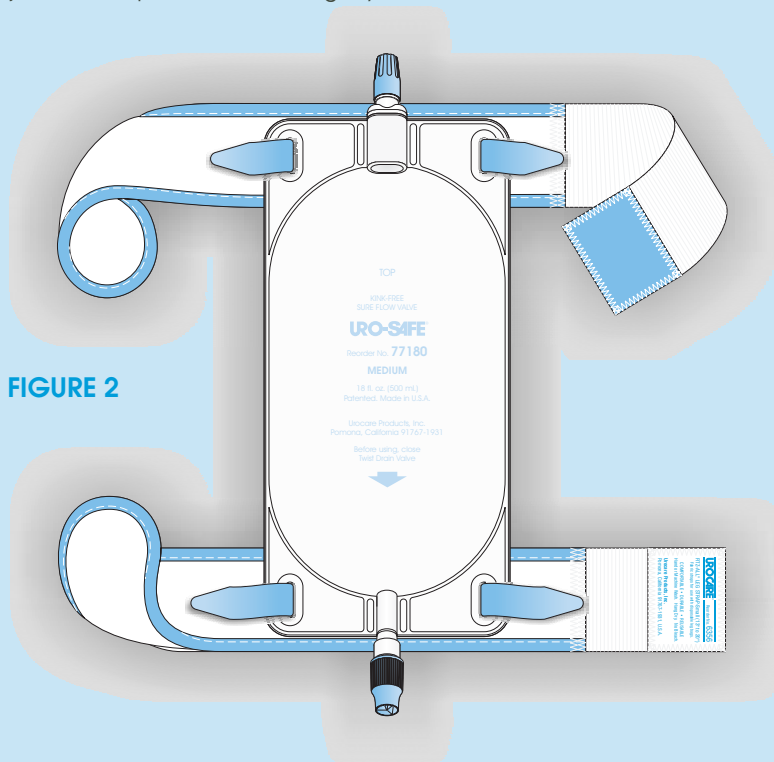


FIGURE 2

KEEPING YOUR FABRIC LEG STRAPS CLEAN:

Close the hook & loop fasteners and join ends of straps as shown in Figure 2. Then, hand or machine wash in warm water. Do not use bleach. Hang dry.

IMPORTANT: The number of times that the hook & loop fasteners are opened and closed determines the life of the fasteners. Do not open and close the fasteners unnecessarily—this will ensure a long life.

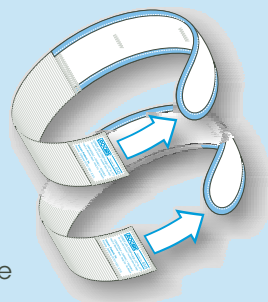


FIGURE 3

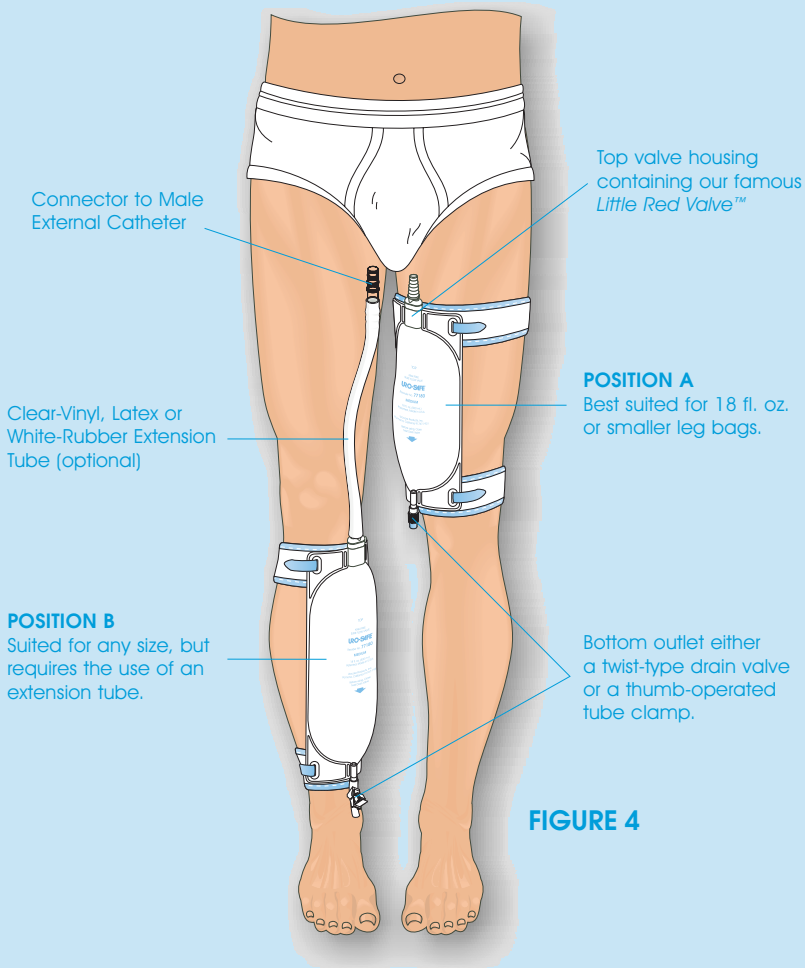


FIGURE 4

Male External Catheters

Each Uro-Cath® and Uro-Con® male external catheters range in size from 25mm to 40mm and are packaged in a window pouch, with or without Urofoam® adhesive foam strips, and/or Uro-Prep® protective skin wipes. Every pouch has detailed, illustrated application instructions on the back side.

Uro-Cath® Molded-Latex Male External Catheters

Because they are formed from a single piece of latex with a funnel-shaped tip design, they are "twist-proof" which allows the fluid pathway to remain open and free from obstructions.



Uro-Con® Texas-Style Male External Catheters


Uro-Con® male external catheters are constructed from three(3) individual pieces (a condom sheath, an insert and a piece of Urocare's pliable, white-rubber tubing in a 2"(5cm) or 4"(10cm) length). Once assembled into a "condom catheter" they become completely collapsible and prevent "pooled-urine", the #1 source of skin irritation and breakdown.




URO-SAFE® VINYL URINARY LEG BAGS

For use with foley indwelling and male external urinary catheters

4 Different Styles, 2 Different Capacities

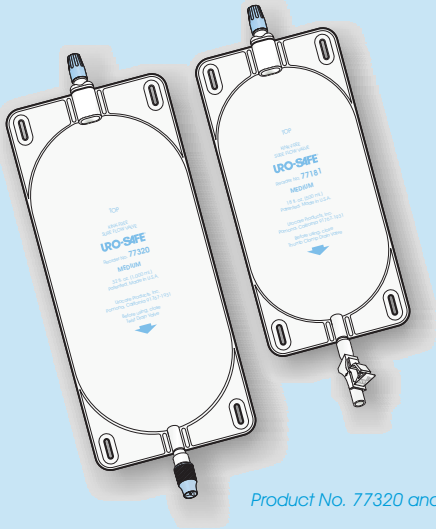

Product No. 77320 and 77181 shown

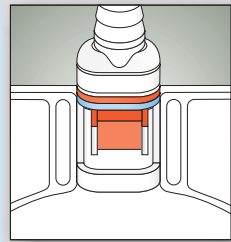

Our famous Little Red Valve™
U.S. Patent number 3,967,645

The Uro-Safe® leg bag has the first ever, built-in, anti-reflux valve that is virtually self-cleaning and interference free... ensuring uninterrupted flow and problem free use.

Uro-Safe® vinyl leg bags are for use with foley indwelling or male external catheters and urostomy drainage. They are available in two sizes: medium or large, two styles: semi-transparent front and back or semi-transparent front with white opaque back and choice of an easy twist-drain valve or a thumb-operated tube clamp on either style. Each Uro-Safe® vinyl leg bag is hand-tested for defects through each phase of the manufacturing process and packaged individually in a sterile, easy-peel open pouch with latex leg straps (No. 60062).

Sold: 12 bags per box

PRODUCT NUMBER	LEG BAG DESCRIPTION	DIMENSIONS*		CAPACITIES*	
		INCHES (")	CENTIMETERS (cm)	Fluid Ounces (fl. Oz.)	Milliliters (ml.)
SEMI-TRANSPARENT FRONT AND WHITE-OPAQUE BACK					
77321	Large with Thumb Clamp	16¾ x 4¾	42.5 x 12.1	32	1,000
77320	Large with Twist Drain Valve	20½ x 3¾	52.4 x 9.5	26	1,000
77181	Small with Thumb Clamp	5¾ x 10	14.6 x 25.0	16	550
77180	Small with Twist Drain Valve	4¾ x 10	12.1 x 25.0	10	550
SEMI-TRANSPARENT FRONT AND BACK					
76321	Large with Thumb Clamp	13¾ x 4¾	34.0 x 12.1	26	1,000
76320	Large with Twist Drain Valve	13 x 3¾	33.0 x 9.5	18	1,000
76181	Small with Thumb Clamp	5¾ x 10	14.6 x 25.0	16	550
76180	Small with Twist Drain Valve	4¾ x 10	12.1 x 25.0	10	550

*Approximate dimensions include adaptors and drain valves. Approximate volumes in fluid ounces and milliliters.



RELATED PRODUCTS

Urocare® Reusable Latex Leg Bags
Uro-Safe® Vinyl Leg Bags
Night Drainage Systems
Itch•ender!® Spray-on Lotion
Uro-Wash® Skin Cleanser
Uro-Con® & Uro-Cath® Male External Catheters
Drain Valves & Adaptors
Drainage Bottles
Uro-Prep® Protective Skin Wipes
Fabric Leg Bag Holders
Fabric Leg Straps
Uro-Bond® II Brush-on Silicone Adhesive
Urofoam® Adhesive Foam Strips
Adhesive Remover Pads
Urolux® Appliance Cleanser & Deodorant
Connectors & Clamps
Extension & Drainage Tubing
Dri-Sleeper™ Nocturnal Enuretic Alarm

For more information on any of Urocare's other products, contact your nearest Urocare dealer or write to:

Urocare Products, Inc.
Customer Service Department
2735 Melbourne Avenue
Pomona, CA 91767-1931, U.S.A.
Tel: (909) 621-6013 • Fax: (909) 621-4436
email: sales@urocare.com
Web: www.urocare.com



PLEASE RECYCLE
Printed on
Recycled Paper