



## FITZ-ALL® WITH BUTTONS URINARY LEG BAG SUPPORT KIT

FOR USE WITH MOST DISPOSABLE  
VINYL URINARY LEG BAGS

**Product No. 6380**



- COMFORTABLE
- WASHABLE
- DURABLE & REUSABLE

ELIMINATES BRUISING, PRESSURE SORES, PULLED HAIR, CIRCULATION RESTRICTION AND BURNS CAUSED BY RUBBER STRAPS.

ADJUSTABLE SELF-GRIPPING FASTENERS.  
APPLICATION IS SAFE AND SIMPLE WITH A GENTLE TOUCH.

### **KIT CONTENTS:**


- 1 Fitz-All® with buttons, upper calf strap (6374)
  - 1 Fitz-All® with buttons, lower calf strap (6375)
- Illustrated use and care instructions inside.

## IMPORTANT! READ THIS FIRST:

To avoid possible drainage problems, read all instructions and precautions carefully before use. Only allow a urinary drainage bag to fill to its maximum capacity in emergency situations. Do not over-tighten straps; this can cause poor circulation and skin breakdown; however, a bag strapped too loosely can slip down the leg and cause damage to male external or foley catheters by stretching them.

## DIRECTION FOR USE:


1. Push the buttons of the straps through the bag eyelets as shown in figure 1.
2. Strap the bag to the leg in the desired position, usually on the inside-bottom of the right or left leg—see figure 3. Make certain that the bag is in an up-right position with the drain valve at the bottom.
3. Adjust straps. Do not over-tighten to avoid circulation restriction.
4. Close bottom drain valve or tube clamp before use.

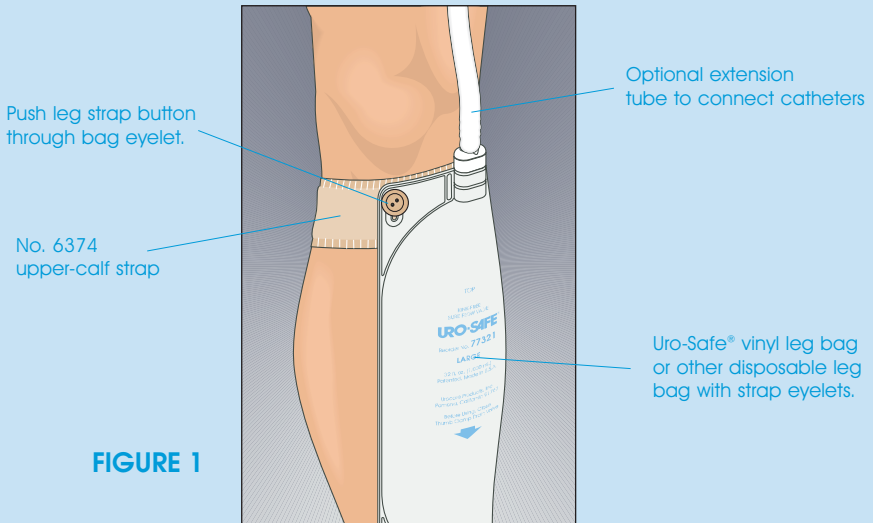


## KEEPING YOUR FABRIC LEG STRAPS CLEAN:

Close the hook & loop fasteners and join ends of straps as shown in Figure 2. Then, hand or machine wash in warm water. Do not use bleach. Hang dry.

**IMPORTANT:** The number of times that the hook & loop fasteners are opened and closed determines the life of the fasteners. Do not open and close the fasteners unnecessarily—this will ensure a long life.

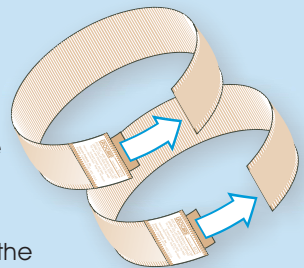

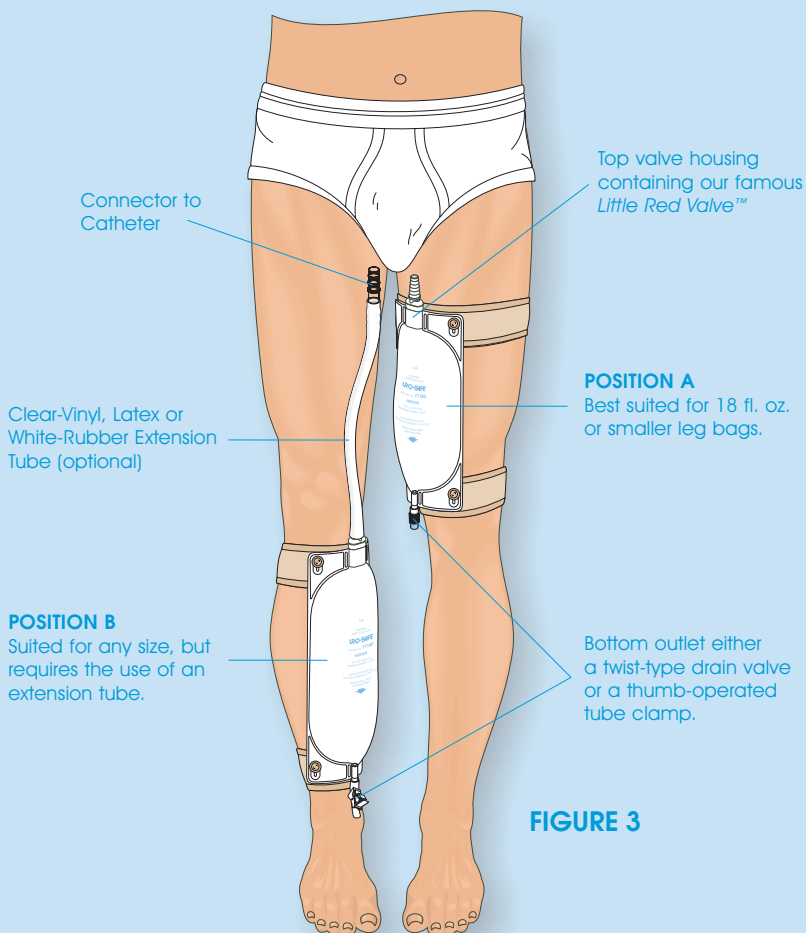

**FIGURE 3**

## Male External Catheters

Each Uro-Cath® and Uro-Con® male external catheters range in size from 25mm to 40mm and are packaged in a window pouch, with or without Urofoam® adhesive foam strips, and/or Uro-Prep® protective skin wipes. Every pouch has detailed, illustrated application instructions on the back side.

### Uro-Cath® Molded-Latex Male External Catheters


Because they are formed from a single piece of latex with a funnel-shaped tip design, they are "twist-proof" which allows the fluid pathway to remain open and free from obstructions.



Product No.  
520135 Shown

### Uro-Con® Texas-Style Male External Catheters

Uro-Con® male external catheters are constructed from three(3) individual pieces (a condom sheath, an insert and a piece of Urocare's pliable, white-rubber tubing in a 2"(5cm) or 4"(10cm) length). Once assembled into a "condom catheter" they become completely collapsible and prevent "pooled-urine", the #1 source of skin irritation and breakdown.

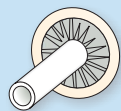

Product No.  
510235 Shown



## RELATED PRODUCTS

Urocare® Reusable Latex Leg Bags  
Uro-Safe® Vinyl Leg Bags  
Night Drainage Systems  
Itch•ender!® Spray-on Lotion  
Uro-Wash® Skin Cleanser  
Uro-Con® & Uro-Cath® Male External Catheters  
Drain Valves & Adaptors  
Drainage Bottles  
Uro-Prep® Protective Skin Wipes  
Fabric Leg Bag Holders  
Fabric Leg Straps  
Uro-Bond® II Brush-on Silicone Adhesive  
Urofoam® Adhesive Foam Strips  
Adhesive Remover Pads  
Urolux® Appliance Cleanser & Deodorant  
Connectors & Clamps  
Extension & Drainage Tubing  
Dri-Sleeper™ Nocturnal Enuretic Alarm

For more information on any of Urocare's other products, contact your nearest Urocare dealer or write to:

Urocare Products, Inc.  
Customer Service Department  
2735 Melbourne Avenue  
Pomona, CA 91767-1931, U.S.A.  
Tel: (909) 621-6013 • Fax: (909) 621-4436  
email: [sales@urocare.com](mailto:sales@urocare.com)  
Web: [www.urocare.com](http://www.urocare.com)



PLEASE RECYCLE  
Printed on  
Recycled Paper