

PI6390-69601100

Tel: 909/621-6013 • Fax: 909/621-4436



Urocare Products, Inc.  
 2735 Melbourne Avenue  
 Pomona, California 91767-1931  
 U. S. A.

For more information on any of Urocare's other products,  
 contact your nearest Urocare dealer or write to:



*Health Care Products*



*Health Care Products*



## UNIVERSAL FABRIC LEG STRAPS

For use with most vinyl, latex and rubber urinary leg bags

- Provides a superior fit over standard latex leg straps
- Fully adjustable self-gripping fasteners make application safe and simple
- Elastic sections make them flexible and comfortable
- Durable, reusable and hand or machine washable

Reorder No. **6390**

One Pair

Fits 8" thru 24" (20 cm thru 61 cm)

## READ THIS FIRST:

For best results, take certain precautions before using these straps. Be certain that there is no loose stitching or other irregularities before each use.

Those with known allergic reactions to certain materials (especially neoprene rubber) should not wear these straps. Discontinue use at first signs of dermatitis or any other type of skin irritation and contact a physician immediately.

Do not over-tighten straps; products made with elastic and/or neoprene rubber material can—if improperly fit—cause circulation restriction or skin breakdown; however, a bag strapped too loosely can slip down the leg and cause damage to male external or foley catheters by stretching them. Should any discomfort occur while wearing, discontinue use immediately and contact a fitter or physician.


To avoid possible drainage problems, read all instructions and precautions carefully before use. Only allow a urinary drainage bag to fill to its maximum capacity in emergency situations.

## APPLICATION INSTRUCTIONS:

1. Be certain that the bottom drain valve or thumb clamp is closed to prevent spills.
2. Wrap the straps around the leg on either the thigh or calf. If the straps are too long, trim by cutting only the elastic between the separations in the loop to achieve the desired length.
3. Thread the end of the strap with the Urocare® logo through the leg bag eyelets so that the straps run behind the leg bag as shown in Figure 1.
4. Place the leg bag in the desired location on the leg and fasten the ends of the straps together—the straps should be snug yet comfortable.

## WASHING INSTRUCTIONS:

Hand or machine wash, warm water, no bleach. Hang dry.

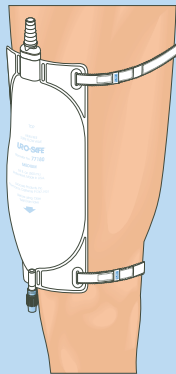

**FIGURE 1**