

UROLUX® is a concentrated solution scientifically formulated to clean and deodorize rubber or plastic urinary and ostomy appliances. Crystal build-ups are dissolved and unpleasant odors are eliminated. Do not use vinegar, bleaches or toilet bowl cleaners to clean your urinary or ostomy appliance; these chemicals will harden and/or distort rubber, damage plastics and cause unpleasant or embarrassing odors.

DIRECTIONS: To thoroughly clean the inside of a soiled appliance: Squeeze ½ ounce (15 mL.) of Urolux® solution into appliance, add one cup (250 mL.) of warm water and agitate contents several times during a 15 minute soaking. Pass cleaning brush (#7003) through all adaptors and plastic parts—**DO NOT PASS BRUSH THROUGH THE RED ANTI-REFLUX VALVE.** Flush contents of appliance with water. Wash outside of appliance with a mild hand soap, rinse and towel dry. Regulate the strength of the Urolux® solution and the cleaning time by the condition of the soiled appliance.

PRECAUTIONS: Urolux® contains phosphoric acid which may cause burns to skin and eyes. Avoid contact with all materials other than glass, plastics, rubber and latex. Avoid contamination of food and contact with skin and eyes. Use only as directed. Keep out of reach of children.

SUGGESTED FIRST AID: SKIN CONTACT: Wash thoroughly with water. EYE CONTACT: Flush thoroughly with water for 15 minutes. IF SWALLOWED: Do not induce vomiting. Drink large amounts of milk or water and contact a physician immediately. SPILL: Flush area with water. Absorb larger spills with a commercial absorbent, collect in covered metal drum for disposal.

IN CASE OF EMERGENCY CALL: 1-800-457-4280.

STORAGE & DISPOSAL: Store in a cool place out of reach of children. Flush bottle thoroughly with water before recycling. **DO NOT REUSE EMPTY BOTTLE.**


Questions/Comments? Call: 1-800-423-4441 or visit us on the web at www.urocare.com

Manufactured by:

Urocare Products, Inc.
Pomona, CA 91767, U.S.A.



UPC-A

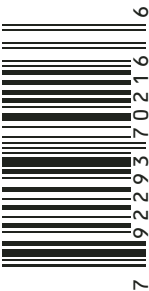

LEARN MORE



UROCARE®
UROCARE PRODUCTS, INC.



#700216

UROLUX®
Urinary & Ostomy Appliance
Cleanser & Deodorant

CONCENTRATED FORMULA

Directions, Precautions &
Antidote Instructions
on Back Panel

DANGER: Harmful or fatal if swallowed. USE ONLY AS DIRECTED. Read all precautions and warnings on back panel carefully before use.

KEEP OUT OF REACH OF CHILDREN

16 fl. oz. (473 mL.)