

Providing Quality Health Care Products Since 1975

# **2018 PRODUCT CATALOG**

INCONTINENCE UROLOGY OSTOMY & TRACHEOSTOMY PERSONAL HYGIENE

LEARN MORE

visit us on the web at <u>www.urocare.com</u>



## About Urocare

## Quality Policy

At Urocare Products we are dedicated to delivering the highest standard of quality in medical products and each member of the Urocare team is committed to making a difference in the lives of people who use our products. Urocare is committed to the development and supply of premium quality health care products to the global community with minimal impact on the environment. Our mission is to provide products that continuously meet or exceed our customer's requirements and to support management's confidence that the highest level of quality is being achieved and sustained. Urocare's commitment to local suppliers and shared compliance with a solid quality program is essential to the attainment of its mission.

### A Brief Company History

Urocare Products was established in 1975 in the Los Angeles area of Southern California and continues to manufacture all of it's products in the U.S.A. with over 95% locally sourced and supplied materials. Urocare has committed its resources to creating innovative products that meet the needs of health care providers and patients alike. Urocare has fast become a name synonymous for quality health care products. Over the years, Urocare has turned a single product into an entire line of health care products that have become recognized and requested in countries around the world. These products have unique practical benefits which are the result of years of research, development, customer feedback and refinement. Each product is designed to be durable, safe and the most easily used product available today.

Urocare is the sole manufacturer of reusable latex leg bags, disposable vinyl leg bags, urinary drainage systems and kits that incorporate the full advantages of our famous, Little Red Valve.<sup>™</sup> This remarkable "High-Flow" anti-reflux valve is self-cleaning and virtually interference free; preventing back flow of urine, thereby reducing the risk of bladder distention and possible infections often incurred with the use of other previous products.



Urocare's focus is on improving the lives of the patients who require our products. We are proud of our long history of achieving this goal, believe our products make a difference, and we are relentless in our efforts to provide the highest quality products and services.

If you believe there is room for improvement upon a Urocare product that you use, we invite you to share your feedback with us. Please visit Urocare's website at <a href="http://www.urocare.com/EN/Contact/FeedbackEN.php">www.urocare.com/EN/Contact/FeedbackEN.php</a>, for any feedback you might want to leave.

### How To Reach Us

Our knowledgeable customer service representatives are available to answer all your incontinence related questions. If we are unable to provide an effective solution to your incontinence needs, we'll refer you to someone who can! Urocare's Customer Service Department is open Monday through Friday from 7:30 AM to 4:00 PM (PST). If you are unable to reach us during these hours, please email or fax your request. To expedite the handling of correspondence, please direct all inquires to the Customer Service Department address below:

Urocare Products, Inc. Customer Service Dept. 2735 Melbourne Avenue Pomona, CA 91767-1931 U.S.A. Toll-Free: **1-800-423-4441** • Tel: 1-909-621-6013 Fax: 1-909-621-4436 Customer Service email: <u>cust.support@urocare.com</u> Sales email: <u>sales@urocare.com</u> Website: <u>www.urocare.com</u>

## Table of Contents

| About Urocare   |
|---|
| Table of Contents   |
| Urine Collection Bags and BottlesUro-Kit® Latex Leg Bag Systems1Urocare® Reusable Latex Leg Bags2Uro-Safe® Disposable Vinyl Leg Bags3Urocare® Urinary Drainage Bag4Urocare® Urinary Drainage Bottle5Urine Collection Bags and Bottles Replacement6, 7 & 8 |
| Male Urinal Systems and Kits  |
| Urocare <sup>®</sup> Male Urinal Systems with Reusable<br>Latex Leg Bag   |
| Personal Hygiene  |
| Urocare <sup>®</sup> "High-Quality" Finger Cots   |
| Straps and Bag Holders  |
| Urocare <sup>®</sup> Anti-Snoring Strap   |
| Urocare <sup>®</sup> Fabric Leg Bag Straps for Reusable<br>Latex Leg Bags   |
| Vinyl Leg Bags  |
| Catheter Strap19Urocare® Universal Fabric Leg Straps19Urocare® Foley® Catheter/Tubing Straps20Urocare® Latex and Latex-Free leg Straps21  |
| Male External Catheters   |
| Uro-Cath <sup>®</sup> "Molded-Latex" Catheters  |

| Enuresis (Bed-Wetting) Treatment Systems<br>DRI Eclipse <sup>®</sup> "Wireless" Enuresis Alarm   |
|--|
| Uro-Bond <sup>®</sup> Brush-on Silicone Adhesive   |
| Skin Care Products<br>Uro-Prep <sup>®</sup> Protective Skin Wipes32  |
| Cleansers and Brushes<br>Urolux <sup>®</sup> Urinary and Ostomy Appliance Cleanser<br>and Deodorant  |
| Drainage and Extension Tubing<br>Clear-Vinyl Drainage and Extension Tubing34<br>White-Rubber/Silicone Elastomer Extension<br>and Drainage Tubing25, 34 |
| Tubing Connectors and ClampsTubing Clamps8 & 35Catheter Connectors35Tubing Connectors35  |
| Drain Valves and AdaptorsUrocare® Quick Drain Valve™   |
| HCPCS Code Reference List.       39         Numerical Product Index       40   |

#### What Are QR codes?

They look like this:




**QR** is short for **Q**uick **R**esponse (they can be read quickly by mobile devices equipped with a camera like a mobile phone or tablet PC). They are used to take information in printed form and transfer it into your mobile device. Once it's scanned into your mobile device, a QR code can give you details about a business, show you a URL which you can click to see detailed product information, watch a product demo or even give you a coupon which you can use at a local business.

A key feature of QR Codes is that instead of requiring a bulky hand-held scanner to scan them, many mobile devices equipped with a camera can scan them. All your mobile device needs is a QR code reader app available for fee at most app stores. It takes literally just a minute to find and install the reader app. Once installed, just open the app, scan the QR code and you're done. It's that simple.

## Urine Collection Bags & Bottles

Uro-Kit<sup>®</sup> Latex Leg Bag Systems

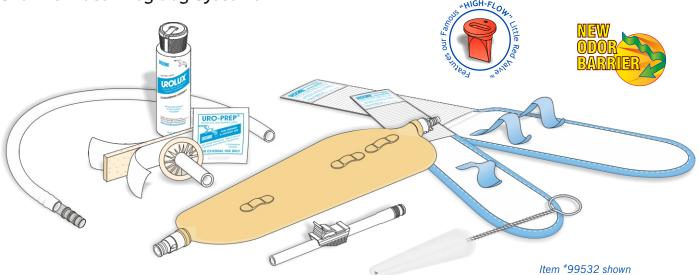

The Uro-Kit<sup>®</sup> system is an ideal introductory kit for those who are new to the problems of urinary incontinence management. Each kit contains all the necessary items and accessories for the control and management of urinary incontinence from the male external catheter right down to the urine collection bag.

#### **CONTENTS:**

- 1 Urocare<sup>®</sup> Seamless Latex Leg Bag<sup>†</sup>
- 1 Urocare<sup>®</sup> Fabric Leg Strap Kit<sup>‡</sup>
- 1 Bottle of Urolux<sup>®</sup> Urinary Appliance Cleanser (#700204)
- 1 Parts Cleansing Brush (#7003)
- 1 Straight-Thru Adaptor with Thumb Clamp (#6005)
- 1 White-Rubber Extension Tubing, 18" Long with Connector (<sup>#</sup>6016)

14 Uro-Con<sup>®</sup> Catheters with Urofoam<sup>®</sup>-2 and Uro-Prep<sup>®</sup> (#5122535 default)\* Illustrated Use and Care Instructions <sup>†</sup> Please refer to the chart below for specific kit contents.

Sold by the box, 1 kit per box. This item is typically made to order. Non-Sterile, Contains Latex

Description and Major Components List<sup>†</sup> Item No.

Extra-Large, contains leg bag \*9544 & leg strap kit \*6350 99544 Large, contains leg bag #9532 & leg strap kit #6350 99532 Long-Slim, contains leg bag <sup>#</sup>9526 & leg strap kit <sup>#</sup>6350 99526 99516\* Sport-Right, contains leg bag #9516 & sport strap #6342 99510\* Sport-Right, contains leg bag #9510 & sport strap #6342 Large-Oval, contains leg bag #8532 & strap kit #6350 98532 Short-Wide, contains leg bag #8526 & strap kit #6350 98526 98518 Medium, contains leg bag \*8518 & strap kit \*6350 Sport-Left, contains leg bag #8516 & sport strap #6342 98516\* Sport-Left, contains leg bag #8510 & sport strap #6342 98510\* 98509 Small, contains leg bag #8509 & leg strap #6344

\* Sport Bags are worn horizontally across the leg beneath shorts, skirts and swimsuits. † See individual items for descriptions, sizes/dimensions, capacities and other details. *‡* Approximate volumes in fluid ounces (fl. oz.) and milliliters (mL)

- \* Please indicate style and millimeter size of male external catheters when ordering—refer to pages 23 & 24 for available styles and sizes.
- <sup>‡</sup> In addition, please specify the size of the fabric leg strap kit—refer to page 14 for individual strap sizes and kit contents.

Capacities<sup>\*</sup>





Case Item No.

| Capacitico            |               |
|-----------------------|---------------|
| 44 fl. oz. (1,300 mL) | Not Available |
| 32 fl. oz. (1,000 mL) | Not Available |
| 26 fl. oz. (800 mL)   | Not Available |
| 16 fl. oz. (500 mL)   | Not Available |
| 10 fl. oz. (300 mL)   | Not Available |
| 32 fl. oz. (1,000 mL) | Not Available |
| 26 fl. oz. (800 mL)   | Not Available |
| 18 fl. oz. (550 mL)   | Not Available |
| 16 fl. oz. (500 mL)   | Not Available |
| 10 fl. oz. (300 mL)   | Not Available |
| 9 fl. oz. (250 mL)    | Not Available |
|                       |               |

D D

## Urine Collection Bags & Bottles

Urocare<sup>®</sup> Reusable Latex Leg Bags



Item #9532 Shown

This is a stocked item.

Non-Sterile, Contains Latex.

1 Reusable Latex Urinary Leg Bag

1 "High-Flow" Anti-Reflux Valve (#6001)

1 Standard Bottom Drain Valve (#6004)

Illustrated Use and Care Instructions

Sold by the each, 1 bag per package.

1 Standard Top Adaptor (#6002)

CONTENTS:

### **COMPLETE PROTECTION NEW "HIGH-FLOW" VALVE**

New high-flow, self-cleaning, patented anti-reflux valve which prevents back-flow while allowing **fast**, uninterrupted drainage for **problem-free use**. Don't settle for urine reflux or back-pressure caused by inferior flutter valves.

#### Nothing beats Urocare's new high-flow Little Red Valve<sup>™</sup> for performance, reliability and safety.

#### **NEW BUILT-IN ODOR BARRIER**


We don't give odor a chance! We added a special new odor control barrier to the inside & outside of our latex leg bags for double odor protection. This special odor barrier makes Urocare's leg bags virtually ODOR-FREE, not

just odor-resistant; thereby preventing embarrassing odors before they start.



COMPLETE CONFIDENCE Urocare has been committed to providing safe, effective and reliable health care products since 1975. The new features of our popular reusable latex leg bags provide greater performance and confidence than ever before. Isn't that what it's all about- being confident and worry-free so that you can spend your time doing what you like doing most? You deserve only the best when it comes to your health and dignity—choose a family of reusable latex leg bags that you can trust completely—choose an original

UROCARE leg bag. Why settle for anything less?

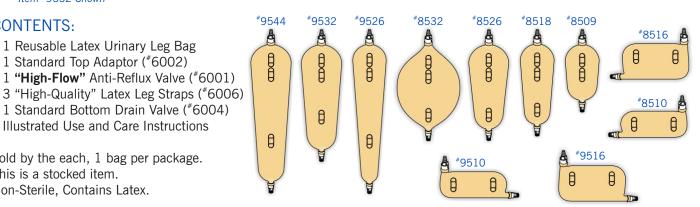

| Item No. | Description | Dimensions <sup>†</sup> |  |
|----------|-------------|-------------------------|--|
| 9544     | Extra-Large | 20.63" x 4.75"          |  |
| 9532     | Large       | 16.75" x 4.75"          |  |
| 9526     | Long-Slim   | 20.63" x 3.75"          |  |
| 9516*    | Sport-Right | 5.75" x 10"             |  |
| 9510*    | Sport-Right | 4.75" x 10"             |  |
| 8532     | Large-Oval  | 13" x 7"                |  |
| 8526     | Short-Wide  | 13.38" x 4.75"          |  |
| 8518     | Medium      | 13" x 3.75"             |  |
| 8516*    | Sport-Left  | 5.75" x 10"             |  |
| 8510*    | Sport-Left  | 4.75" x 10"             |  |
| 8509     | Small       | 9.75" x 3.38"           |  |

44 fl. oz. (1,300 mL) 32 fl. oz. (1,000 mL) 26 fl. oz. (800 mL) 16 fl. oz. (500 mL) 10 fl. oz. (300 mL) 32 fl. oz. (1,000 mL) 26 fl. oz. (800 mL) 18 fl. oz. (550 mL) 16 fl. oz. (500 mL) 10 fl. oz. (300 mL) 9 fl. oz. (250 mL)

Capacities<sup>‡</sup>







\* Sport Bags are worn horizontally across the leg beneath shorts, skirts and swimsuits.

*†* Approximate dimensions in inches(") and centimeters(cm) including adaptors and drain valves.

*‡* Approximate volumes in fluid ounces (fl. oz.) and milliliters (mL)

Urine Collection Bags & Bottles

Uro-Safe<sup>®</sup> Disposable Vinyl Leg Bags

D

300

- 200

- 100 Approx

# THE FIRST SIGNIFICANT IMPROVEMENT IN URINARY LEG BAGS IN OVER 50 YEARS!

Uro-Safe<sup>®</sup> bags have the first ever, built-in, anti-reflux valve that is virtually **selfcleaning** and **interference free**... ensuring uninterrupted flow and problem free use.

Because our anti-reflux valve is enclosed in a hard valve housing, it can't be held shut even when in sitting or reclining positions unlike other inferior products.

Uro-Safe® Why settle

for anything less?

ATEX-FREE

**STERILE (EO)** 

HIGH-FL



Easy, Twist-Type

**Drain Valve Closure** 

MADE IN



Improved, Reinforced Strap Eyelets



Two-Position Thumb Clamp Closure

LEARN MORE

#### CONTENTS:

C

77180

MEDIUM

LATEX-FREE

- 1 Uro-Safe<sup>®</sup> Disposable Vinyl Urinary Leg Bag<sup>†</sup> with
- 1 Pair of "Latex-Free" Leg Straps and Buttons (#60062)

Item #77180 Shown

- Illustrated Use and Care Instructions
- <sup>†</sup> Please refer to the chart below for specific box contents.

Sold: 1 bag with straps per package/12 packages per case.\* This is a stocked item. Sterile (EO), Latex-Free.

#### URO-SAFE® BAGS WITH SEMI-TRANSPARENT FRONT AND OPAQUE-WHITE BACK

| Item No. | Description             | Dimensions <sup>†</sup> | Capacities <sup>‡</sup> | Case Item No.   |
|----------|-------------------------|-------------------------|-------------------------|-----------------|
| 77321*   | Large with Thumb Clamp  | 16.5" x 5.25"           | 32 fl. oz. (1,000 mL)   | 7732112 (12/CS) |
| 77320*   | Large with Twist Drain  | 15" x 5.25"             | 32 fl. oz. (1,000 mL)   | 7732012 (12/CS) |
| 77181*   | Medium with Thumb Clamp | 14.38" x 4.75"          | 18 fl. oz. (550 mL)     | 7718112 (12/CS) |
| 77180*   | Medium with Twist Drain | 13" x 4.75"             | 18 fl. oz. (550 mL)     | 7718012 (12/CS) |
|          |                         | S WITH SEMI-TRANSPAR    | RENT FRONT AND BACK     |                 |
| Item No. | Description             | Dimensions <sup>†</sup> | Capacities <sup>‡</sup> | Case Item No.   |
| 76321*   | Large with Thumb Clamp  | 16.5" x 5.25"           | 32 fl. oz. (1,000 mL)   | 7632112 (12/CS) |
| 76320*   | Large with Twist Drain  | 15" x 5.25"             | 32 fl. oz. (1,000 mL)   | 7632012 (12/CS) |
| 76181*   | Medium with Thumb Clamp | 14.38" x 4.75"          | 18 fl. oz. (550 mL)     | 7618112 (12/CS) |
| 76180*   | Medium with Twist Drain | 13" x 4.75"             | 18 fl. oz. (550 mL)     | 7618012 (12/CS) |
|          |                         |                         |                         |                 |

\* These items are not sold individually, sold only by the case.


*†* Approximate dimensions in inches(") and centimeters(cm) including adaptors and drain valves.

‡ Approximate volumes in fluid ounces (fl. oz.) and milliliters (mL)

## Urine Collection Bags & Bottles

### Urocare<sup>®</sup> Urinary Drainage Bag

The Urocare Urinary Drainage Bag is made from soft vinyl with a transparent front and white-opaque back. It is graduated from 100 mL to 2,000 mL in 100 mL increments and for smaller amounts, a second scale is provided from 25 mL to 100 mL in 25 mL increments. Features a double reinforced hanger strap, "T"-Valve for quick and easy draining, 50" (127 cm) of pliable clear-vinyl tubing with a graduated nozzle to allow for multiple sized attachments, a breather patch to allow free flow of urine, a bed sheet clip, needleless sample port as well as being 100% Latex-Free.

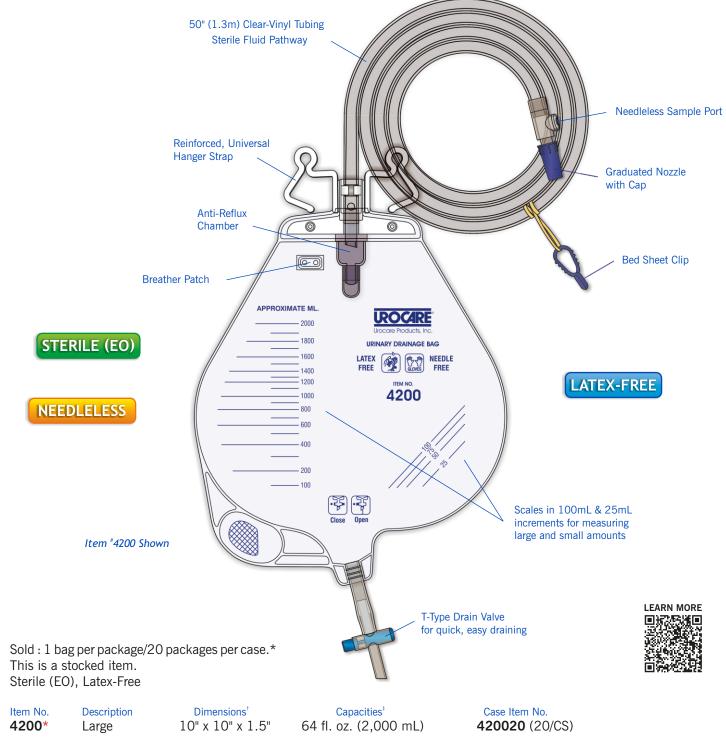

\* These items are not sold individually, only by the case.

† Approximate dimensions in inches(") and centimeters(cm) including adaptors and drain valves.


*‡* Approximate volumes in fluid ounces (fl. oz.) and milliliters (mL)

## Urine Collection Bags & Bottles

## Urocare<sup>®</sup> Urinary Drainage Bottle

An effective bedside drainage system that can be used with Foley<sup>®</sup> indwelling or male external catheters, urostomy, ileostomy, suprapubic applications etc. Made from rigid, high-density polyethylene (HDPE) it's durable and reusable. It is graduated from 300 mL to 2,000 mL in increments of 100 mL


The risk of tipping or being pulled over is greatly reduced by a large bottom surface area with a small corner radius. Conveniently and simply transferred from one side of the bed to the other by use of the large finger loops and hooks for hanging. A large opening makes for easy cleaning and emptying.

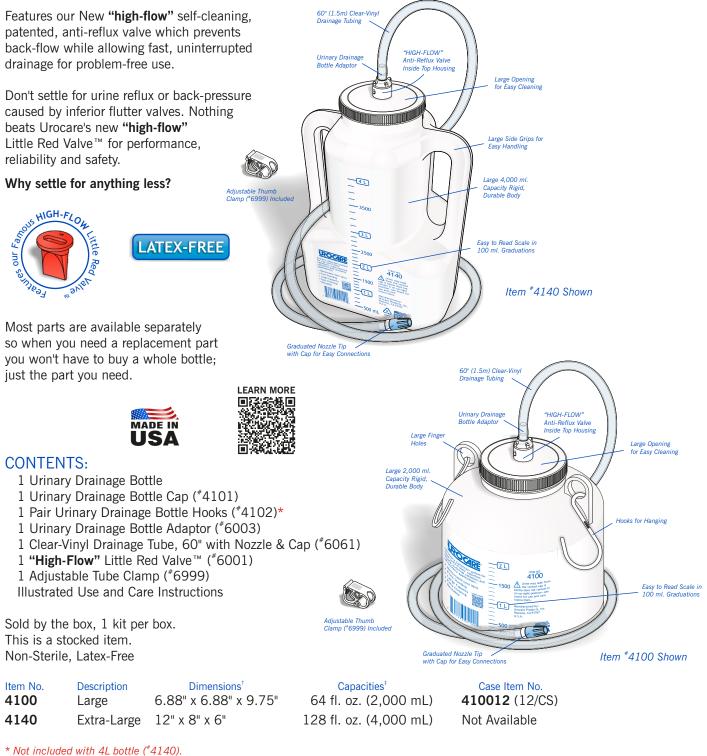

*†* Approximate dimensions in inches(") and centimeters(cm) including adaptors.

*‡* Approximate volumes in fluid ounces (fl. oz.) and milliliters (mL)

Product Catalog 2018 • Urocare Products, Inc.

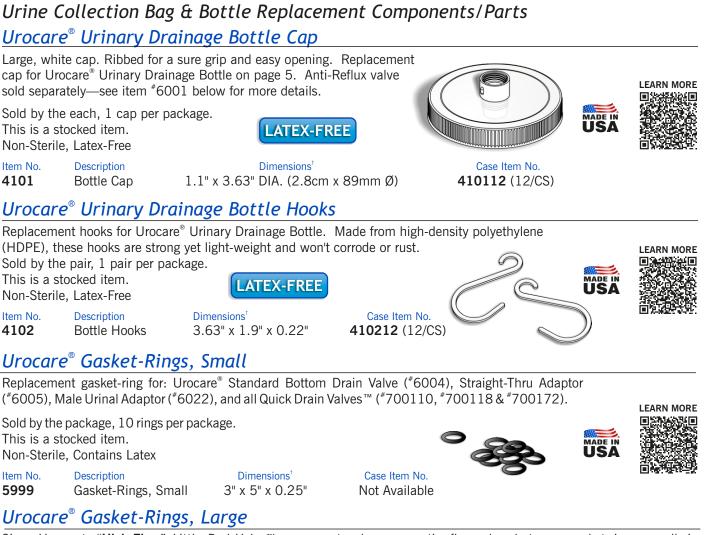

Since Urocare's "**High-Flow**" *Little Red Valve*<sup>™</sup> serves not only as an anti-reflux valve, but as a gasket-ring as well, in circumstances where the valve is not used, this gasket-ring must be used in place of the valve to prevent leaks in all Urocare<sup>®</sup> Reusable Latex Leg Bags and Urocare<sup>®</sup> Urinary Drainage Bottle which incorporate this valve.

Sold by the package, 10 rings per package. This is a stocked item. Non-Sterile, Contains Latex

Urine Collection Bags & Bottles

Item No.Description6000Gasket-Rings, Large

, Large 3" x 5" x 0.25"





Urocare<sup>®</sup> "High-Flow" Little Red Valve™

This new, **high-flow**, **self-cleaning**, patented, anti-reflux valve **prevents back-flow** while allowing fast, uninterrupted drainage for problem-free use. Don't settle for urine reflux or back-pressure caused by inferior flutter valves. Nothing beats Urocare's new **"High-Flow"** *Little Red Valve*<sup>™</sup> for performance, reliability and safety. Replacement anti-reflux valve for use in all Urocare<sup>®</sup> Reusable Latex Leg Bags and Urocare<sup>®</sup> Urinary Drainage Bottle.



*†* Approximate dimensions in inches(") and centimeters(cm).

Quality Health Care Products Since 197

## Urine Collection Bags & Bottles

### Urine Collection Bag & Bottle Replacement Components/Parts

### Urocare<sup>®</sup> Standard Top Adaptor

Features a graduated top with reverse-grip that provides a secure connection for multiple tubing diameters. Replacement top adaptor for use with all Urocare<sup>®</sup> Reusable Latex Leg Bags.

Sold by the each, 1 adaptor per package. This is a stocked item. Non-Sterile, Latex-Free.

Item No.Description6002Standard Top Adaptor

Dimensions<sup>†</sup> 1.24" x 0.84" (3.15cm x 2.13cm)





Case Item No. 600210 (10/CS)

### Urocare<sup>®</sup> Urinary Drainage Bottle Adaptor

Provides a secure connection for Urocare<sup>®</sup> Clear-Vinyl Drainage Tubing. Replacement adaptor for use with Urocare<sup>®</sup> Urinary Drainage Bottle. May also be used with Urocare<sup>®</sup> Reusable Latex Leg Bags as a replacement for the Standard Top Adaptor (<sup>#</sup>6002) for those who use clear-vinyl tubing.

ATEX-FREE

Dimensions<sup>†</sup>

**ATEX-FREE** 

Sold by the each, 1 adaptor per package. This is a stocked item. Non-Sterile, Latex-Free.

Item No.Description6003Drainage Bottle Adaptor

otor 1.24" x 0.84" (3.15cm x 2.13cm)





Case Item No. 600310 (10/CS)

### Urocare<sup>®</sup> Standard Bottom Drain Valve

Features sealed threads so fingers don't come in contact with urine while draining. Only requires 1½ turns to open or close. Replacement drain value for use with all Urocare<sup>®</sup> Reusable Latex Leg Bags and Urocare<sup>®</sup> Male Urinal Sheaths. Includes Small Black Gasket-Ring (<sup>#</sup>5999) preassembled.

Sold by the each, 1 valve per package. This is a stocked item. Non-Sterile, Latex-Free.

Item No. Description 6004 Standard Bottom Drain Valve Dimensions<sup>†</sup>

1.53" x 0.6" (3.88cm x 1.52cm)

ATEX-FREE





Case Item No. 600410 (10/CS)

### Urocare<sup>®</sup> Latex Leg Strap Replacement Set

These straps are replacements for those normally packaged with Urocare<sup>®</sup> Reusable Latex Leg Bags or most other latex or rubber leg bags up to 44 fl. oz. (1,300 mL). Includes white plastic buttons. Straps may also be used with most disposable vinyl leg bags.\*

Sold by the set, 3 straps with buttons per package. This is a stocked item.

Non-Sterile, Contains Latex.

Item No.Description6006Latex Leg Straps with Buttons Set





600610 (10 Sets/CS)

#### Urocare<sup>®</sup> "Latex-Free" Leg Straps

Replacement straps for those normally packaged with Uro-Safe<sup>®</sup> disposable vinyl leg bags or most other disposable urine collection bags. Includes white plastic buttons.

Suitable for bags up to 32 fl. oz. (1 L).\*

Sold by the set, 2 straps with buttons per package. This is a stocked item. Non-Sterile, Latex-Free.

Item No.Description60062Latex-Free Leg Straps with Buttons

*†* Approximate dimensions in inches(") and centimeters(cm). \* Bags must have eyelets for straps to pass through.



## Urine Collection Bags & Bottles

### Urine Collection Bag & Bottle Replacement Components/Parts

### Urocare<sup>®</sup> Clear-Vinyl Drainage Tubing

Used to connect male external or Foley<sup>®</sup> indwelling catheters to bedside urinary drainage bags, reusable or disposable leg bags, urinary drainage bottles or other urine collection/drainage appliances. Provides a snug fit on most standard sized connectors, adaptors and fittings while resisting kinks. Can be used as a replacement tube for the non-sterile tubing (\*6061) included in the Urinary Drainage Bottle ( $^{4}4100$ ). Includes graduated adaptor/nozzle with cap.



### Urocare<sup>®</sup> Clear-Vinyl Drainage Tubing

Used to connect male external or Foley<sup>®</sup> indwelling catheters to bedside urinary drainage bags, reusable or disposable leg bags, urinary drainage bottles or other urine collection/drainage appliances. Provides a snug fit on most standard sized connectors, adaptors and fittings while resisting kinks. Replacement tube for the tubing included in the Urinary Drainage Bottle (#4100). Includes graduated adaptor/nozzle with cap.



### Urocare<sup>®</sup> Adjustable Tube Clamp

Provides an incremental, adjustable closure allowing a reduction or complete restriction of fluids passing through clearvinyl, white-rubber, silicone elastomer or latex tubing 0.44" O.D. (1.1cm Ø) or less. Replacement tube clamp found in the Urocare<sup>®</sup> Urinary Drainage Bottle (#4100).

Sold: 1 clamp per package/10 packages per box. This is a stocked item. Non-Sterile, Latex-Free

Item No. Description 6999\* Adjustable Tube Clamp, White **ATEX-FREE** 



**Dimensions**<sup>†</sup> 1.24" x 0.84" (3.15cm x 2.13cm)


Case Item No. 699910 (10/BX) 699950 (50/CS)

\* These items are not sold individually, sold only by the box and/or case.

*†* Approximate dimensions in inches(") and centimeters(cm) including adaptors.

## Male Urinal Systems & Kits

Urocare® Male Urinal Systems with Reusable Latex Leg Bag

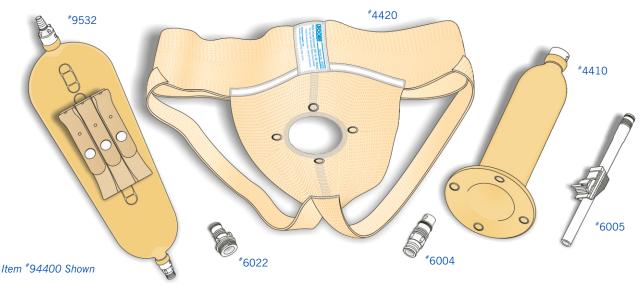

Our Male Urinal System is a **lightweight, comfortable** solution to incontinence management with a **quick and easy application** method that eliminates the need for inconvenient multiple changes, adhesives and adhesive removers.

It's an effective and **economical** daytime alternative to adult under pads and diapers for men that experience "Stress **Incontinence**" (leaks when laughing, coughing or exercising, etc.) or "Overflow Incontinence" (bladder is constantly full, but is unable to release urine resulting in leakage caused by diabetes, an enlarged prostate, side effects of medications, spinal cord or nerve injuries) and dribbling.

It can serve as either a drip urinal for those that are only partially incontinent or with the use of the various supplied adaptors, it can be directly connected to the reusable latex leg bag included with this system. Each component in this system is available separately so when you need a replacement component you won't need to buy a whole new system—just the parts needed.

#### **CONTENTS:**

- 1 Urinal Suspensory Garment, Standard<sup>†</sup>
- 1 Male Urinal Sheath, Large<sup>†</sup>
- 1 Standard Bottom Drain Valve (#6004)
- 1 Male Urinal to Latex Leg Bag Adaptor (#6022)
- 1 Straight-Thru Adaptor with Thumb Clamp (\*6005)
- 1 Urocare Reusable Latex Leg Bag (<sup>#</sup>9532)
- 3 Latex Leg Straps with Buttons (#6006)
- Illustrated Use and Care Instructions

<sup>*†*</sup> Please refer to the chart below for specific kit contents.

Sold by the box, 1 system per box. See chart below for item availability. Non-Sterile, Contains Latex





#### MALE URINAL SYSTEMS WITH UNIVERSAL VELCRO® CLOSURE SUSPENSORY GARMENT

| Item No. | Description | List of Primary Components  | Case Item No. | Availability  |
|----------|-------------|---|---------------|---------------|
| 94403    | Large       | Bag <sup>#</sup> 9532, Garment <sup>#</sup> 4422 & Sheath <sup>#</sup> 4410 | Not Available | Made to Order |
| 94402    | Small       | Bag <sup>#</sup> 9532, Garment <sup>#</sup> 4422 & Sheath <sup>#</sup> 4409 | Not Available | Made to Order |
|          |             |   |               |               |


#### MALE URINAL SYSTEMS WITH STANDARD LACED CLOSURE SUSPENSORY GARMENT\*

| Item No. | Description | List of Primary Components  | Case Item No. |         |
|----------|-------------|---|---------------|---------|
| 94404    | Extra-Large | Bag <sup>#</sup> 9532, Garment <sup>#</sup> 4423 & Sheath <sup>#</sup> 4410 | Not Available |         |
| 94401    | Large       | Bag <sup>#</sup> 9532, Garment <sup>#</sup> 4421 & Sheath <sup>#</sup> 4410 | Not Available | Stocked |
| 94400    | Medium      | Bag <sup>#</sup> 9532, Garment <sup>#</sup> 4420 & Sheath <sup>#</sup> 4410 | Not Available |         |
| 94399    | Small       | Bag <sup>#</sup> 9532, Garment <sup>#</sup> 4420 & Sheath <sup>#</sup> 4409 | Not Available |         |

*†* Please refer to pages 2 and 11 for specific component details.

## Male Urinal Systems & Kits

Urocare<sup>®</sup> Male Urinal Kits

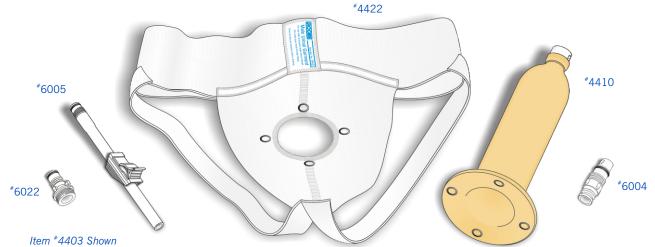

Our Male Urinal System is a **lightweight**, **comfortable** solution to incontinence management with a **quick and easy application** method that eliminates the need for inconvenient multiple changes, adhesives and adhesive removers.

It's an effective and **economical** daytime alternative to adult under pads and diapers for men that experience "Stress **Incontinence**" (leaks when laughing, coughing or exercising, etc.) or "Overflow Incontinence" (bladder is constantly full, but is unable to release urine resulting in leakage caused by diabetes, an enlarged prostate, side effects of medications, spinal cord or nerve injuries) and dribbling.

It can serve as either a drip urinal for those that are only partially incontinent or with the use of the various supplied adaptors, it can be directly connected to most any reusable latex or disposable vinyl leg bag. Each component in this kit is available separately so when you need a replacement component you won't need to buy a whole new kit—just the parts needed.

#### **CONTENTS:**

- 1 Urinal Suspensory Garment, Standard<sup>†</sup>
- 1 Male Urinal Sheath, Large<sup>†</sup>
- 1 Standard Bottom Drain Valve (#6004)
- 1 Male Urinal to Latex Leg Bag Adaptor (#6022)
- 1 Straight-Thru Adaptor with Thumb Clamp (\*6005)
- Illustrated Use and Care Instructions

<sup>†</sup> *Please refer to the list below for specific kit contents.* Sold by the box, 1 system/kit per box.

This is a stocked item. Non-Sterile, Contains Latex







#### MALE URINAL KITS WITH UNIVERSAL VELCRO® CLOSURE SUSPENSORY GARMENT

| Description | List of Primary Components                                      | Case Item No.<br>Not Available   |  |
|-------------|---|--|--|
| 6           |   |  |  |
|             |   |  |  |
| M           | ALE URINAL KITS WITH LACED CLOSU                                | IRE SUSPENSORY GARMENT <sup>*</sup>  |  |
| Description | List of Primary Components                                      | Case Item No.  |  |
| Extra-Large | Garment #4423 & Sheath #4410                                    | Not Available  |  |
| Large       | Garment <sup>#</sup> 4421 & Sheath <sup>#</sup> 4410            | Not Available  |  |
| Medium      | Garment #4420 & Sheath #4410                                    | Not Available  |  |
| Small       | Garment <sup>#</sup> 4420 & Sheath <sup>#</sup> 4409            | Not Available  |  |
|             | Large<br>Small<br>Description<br>Extra-Large<br>Large<br>Medium | LargeGarment *4422 & Sheath *4410SmallGarment *4422 & Sheath *4409MALE URINAL KITS WITH LACED CLOSUDescriptionList of Primary ComponentsExtra-LargeGarment *4423 & Sheath *4410LargeGarment *4421 & Sheath *4410MediumGarment *4420 & Sheath *4410 | LargeGarment #4422 & Sheath #4410Not AvailableSmallGarment #4422 & Sheath #4409Not AvailableMALE URINAL KITS WITH LACED CLOSURE SUSPENSORY GARMENT*DescriptionList of Primary ComponentsCase Item No.Extra-LargeGarment #4423 & Sheath #4410Not AvailableLargeGarment #4421 & Sheath #4410Not AvailableMediumGarment #4420 & Sheath #4410Not Available |

*†* Please refer to page 11 for specific component details.

## Male Urinal Systems & Kits

### Male Urinal Systems Replacement Components/Parts

### Urocare<sup>®</sup> Male Urinal Replacement Sheaths

Replacement sheaths found in Urocare<sup>®</sup> Male Urinal Systems with Reusable Latex Leg Bag and Urocare<sup>®</sup> Male Urinal Kits. May be used with all Urocare<sup>®</sup> male urinal suspensory garments (<sup>#</sup>4422, <sup>#</sup>4421 or <sup>#</sup>4420). Snaps for easy assembly. Incorporates Urocare's "Odor-Barrier" technology.



#### Urocare<sup>®</sup> Male Urinal Replacement Suspensory Garments


Our replacement suspensory garments with standard laced closure are made from cream colored cotton and elastic which is comfortable, light and doesn't promote heavy sweating. Adjustments to suit individual requirements can be made with the laces at the back of the waist band. Replacement garments for those found in Urocare® Male Urinal Systems with Reusable Latex Leg Bag and Urocare<sup>®</sup> Male Urinal Kits. May be used with either size male urinal sheath (\*4409 or \*4410).

#### CONTENTS:

1 Urinal Suspensory Garment<sup>†</sup> Illustrated Use & Care Instructions

<sup>†</sup> Please refer to the list below for specific item contents.

Sold by the each, 1 garment per package. This is a stocked item. Non-Sterile, Contains Latex,

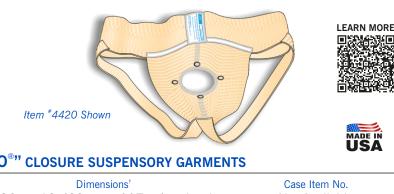

### "UNIVERSAL VELCRO<sup>®</sup>" CLOSURE SUSPENSORY GARMENTS

| Item No. | Description                      | Dimensions' 26" $\sim$ 46" (66cm $\sim$ 117cm) waist size | Case Item No. |
|----------|----------------------------------|---|---------------|
| 4422     | Urinal Suspensory Garment, Large |   | Not Available |
|          | <b>"STANDARD LAC</b>             | ED" CLOSURE SUSPENSORY GARMENTS                           |               |
| Item No. | Description                      | Dimensions <sup>†</sup>                                   | Case Item No. |
| 4423     | Suspensory Garment, Extra-Large  | 46" ~ 54" (117cm ~ 137cm) waist size                      | Not Available |
| 4421     | Suspensory Garment, Large        | 38" ~ 46" (97cm ~ 117cm) waist size                       | Not Available |

Suspensory Garment, Small 4420

### Urocare<sup>®</sup> Gasket-Rings, Small

Replacement gasket-ring for: Urocare® Standard Bottom Drain Valve (#6004), Straight-Thru Adaptor (#6005), Male Urinal Adaptor (<sup>#</sup>6022), and all Quick Drain Valves<sup>™</sup> (<sup>#</sup>700110, <sup>#</sup>700118 & <sup>#</sup>700172).

| This is a sto    | ackage, 10 rings per pac<br>cked item.<br>Contains Latex | kage.                                      | 0                              |     |  |
|------------------|--|--|--------------------------------|-----|--|
| Item No.<br>5999 | Description<br>Gasket-Rings, Small                       | Dimensions <sup>†</sup><br>3" x 5" x 0.25" | Case Item No.<br>Not Available | 690 |  |

*†* Approximate dimensions in inches(") and centimeters(cm).

26"~ 38" (66cm ~ 97cm) waist size

Product Catalog 2018 • Urocare Products, Inc.




Not Available

## Male Urinal Systems & Kits

### Male Urinal Replacement Components/Parts Urocare<sup>®</sup> "High-Flow" Little Red Valve™

This new **high-flow**, **self-cleaning**, patented, anti-reflux valve **prevents back-flow** while allowing **fast**, uninterrupted drainage for **problem-free use**. Don't settle for urine reflux or back-pressure caused by inferior flutter valves. Nothing beats Urocare's new **"High-Flow"** *Little Red Valve*<sup>™</sup> for performance, reliability and safety. Replacement anti-reflux valve for use in all Urocare<sup>®</sup> Reusable Latex Leg Bags and Urocare<sup>®</sup> Urinary Drainage Bottle.

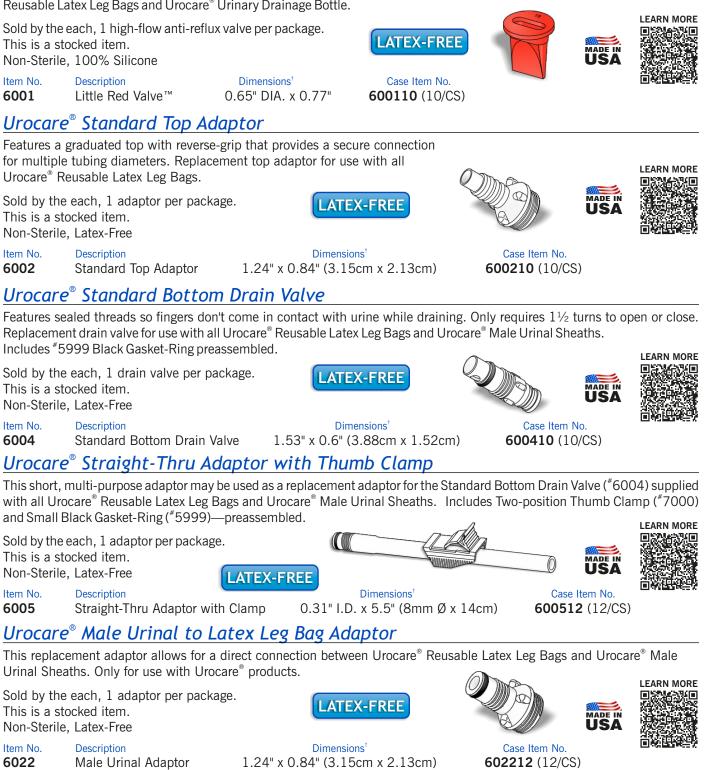

## Personal Hygiene

### Finger Cots

### Urocare<sup>®</sup> "High-Quality" Latex Finger Cots

Finger Cots are typically used by home heath care givers, kitchen workers and others to cover cuts and open wounds while working, in order to prevent infection and the spread of disease. They are very useful for keeping bandages on an injured finger, especially for activities like typing. They can also be used by massage therapists to prevent massage lotions or oils from entering small cuts or abrasions on the fingers. Finger cots are especially important in areas of manufacturing during handling of small parts or components to prevent contamination by humans or where only protection of the fingers is necessary.

These finger cots are tissue thin, but super tough. A "high-quality", white latex assures a smooth finish, good sensitivity and a perfect fit. The best test for good finger cots is that wearing them is like wearing nothing at all. They are also used to keep moisture and dirt out of skin abrasions, cuts or stitches on the fingers.

Sold by the box, 1 gross (144 each) per dispenser box. This is a stocked item. Non-Sterile, Contains Latex.

Item \*500220 Shown

 Item No.
 Description
 Dimensions<sup>†</sup>

 500222
 Large
 22mm DIA. (Ø) x 3" (7.6cm) Long

 500220
 Medium
 20mm DIA. (Ø) x 3" (7.6cm) Long

 500218
 Small
 18mm DIA. (Ø) x 3" (7.6cm) Long

Case Item No. Not Available Not Available Not Available

LEARN MORE

500220

*†* Approximate dimensions in inches(") and centimeters(cm) including adaptors and drain valves.

### Urocare<sup>®</sup> Anti-Snoring Strap

#### What is snoring?


Snoring is a harsh or some times rattling noise that is usually produced during deep sleep. The sound is produced when air inhaled through the mouth vibrates the tissue in the roof of the mouth near the throat. As the soft tissue vibrates, the lips, cheeks, and nostrils can also vibrate, making the sound even louder. Almost everyone snores occasionally, but men usually snore more often than women and children. Everyone knows this problem, as snorers themselves or as involuntarily listeners. In either case, the results are undesirable.

#### Is snoring unhealthy?

An estimated **one in every 30 to 50 adults** has sleep apnea, a condition in which breathing is repeatedly interrupted during sleep. In one form of sleep apnea, called obstructive sleep apnea, the muscles that hold open the airway relax, causing the breathing passage to collapse. When oxygen levels in the blood become too low, the person wakes briefly to take a **gasping breath**, which sounds like a loud snort. People with this condition don't remember these short awakenings and often assume that they have slept through the night. As a result they may not understand why they develop excessive daytime sleepiness.<sup>†</sup>

#### Is Snoring bad for the heart?

Due to a lowered oxygen supply, the heart must work harder. For persons with heart disease, snoring can be dangerous. An increased demand on an unhealthy heart causes the heart to beat quickly

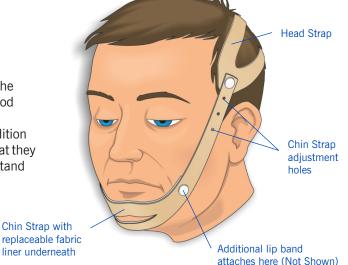

so that it wakens the person sleeping. The person often finds themselves drenched in sweat with a heart beating rapidly.

Excessive palpitation is often attributed to dreaming, but that is not usually the case. The actual cause is snoring.

#### **Healthy sleep**


Healthy, quiet and restful sleep is possible with the Urocare<sup>®</sup> Anti-Snoring Strap. It helps snorers and saves others from torment. It is made from soft, synthetic rubber and padded with a fine cotton and foam laminate. Through gentle training, the lower jaw remains closed during sleep. Every snorer should in their own interest, but also with regard for his fellow man use the Urocare<sup>®</sup> Anti-Snoring Strap.

| 4 Plast  | Strap<br>Strap  |            |               |        |                    |
|--|---|------------|---------------|--------|--------------------|
| Sold by th<br>This is a s<br>Non-Steri<br>Item No. | ne each, 1 set per package.<br>stocked item.<br>le, Latex-Free<br>Description | Dimensions |               | "Silen | ce truly<br>Iden!" |
| 6200   | Anti-Snoring Strap Set  | Universal  | Not Available | :- 60  | laen               |

### Urocare<sup>®</sup> Fabric Leg Bag Holders

#### For use with Reusable Latex or Disposable Vinyl Leg Bags

These leg bag holders are a simple and cost effective solution to a difficult problem. They prevent contact between the leg bag and the skin; this reduces sweating while allowing the skin to breathe freely. Their unique construction provides a simple method for securing a urine collection bag to the leg without the use of leg straps which are often the cause of circulation restriction in some individuals.

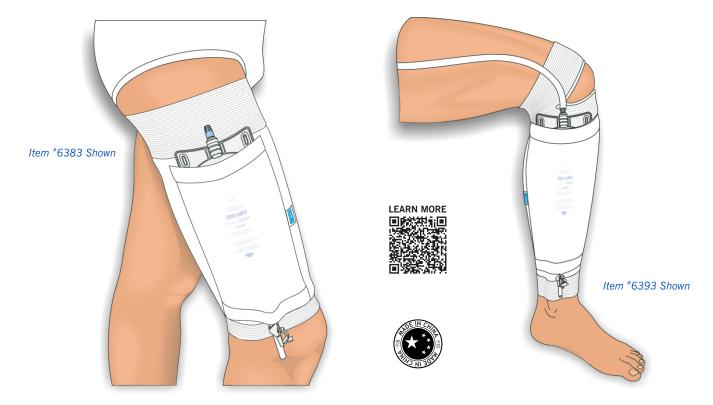

Broad elasticated cotton bands combined with a stretchable fabric ensure the urine bag is secured firmly, but comfortably to the leg. These holders can be worn on the inner or outer side of the leg. Each holder is made from a durable blend of 95% cotton and 5% Lycra that allows them to be hand or machine washed. Each style provides an excellent fit without circulation restriction and is available in multiple sizes—refer to the charts below for sizing information.

Sold by the each, 1 holder per package. This is a stocked item. Non-Sterile, Latex-Free

LATEX-FREE



95% Cotton

|--|

| Dimensions'  |
|--------------|
| 38" (36.5cm) |
| 53" (34.5cm) |
| 53" (32cm)   |
|              |


#### FABRIC LEG BAG HOLDERS FOR THE "UPPER LEG"

| Item No.<br>6384 | Description<br>Upper Leg, Large |
|------------------|---------------------------------|
| 6383             | Upper Leg, Medium               |
| 6382             | Upper Leg, Small                |

Dimensions (Upper Thigh, Lower Thigh)<sup>1</sup> 24" (61cm), 19.25" (49cm) 22.75" (57.7cm), 18.75" (46.7cm) 21.5" (54.4cm), 17.5" (44.3cm) Case Item No. Not Available Not Available Not Available

Case Item No. Not Available Not Available Not Available

Urocare® Fabric Leg Bag Straps for Urocare® Reusable Latex Leg Bags ELIMINATE BRUISING, PRESSURE SORES, PULLED HAIR, CIRCULATION RESTRICTION AND BURNS CAUSED BY RUBBER STRAPS

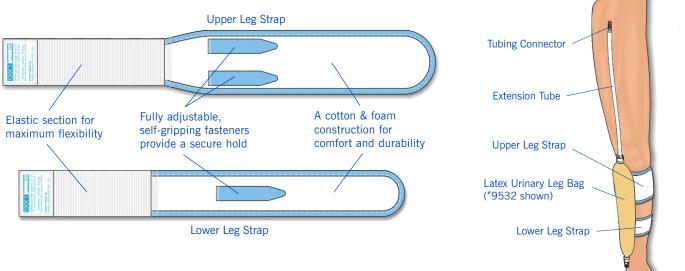

## FULLY ADJUSTABLE, SELF-GRIPPING FASTENERS MAKE APPLICATION SAFE AND SIMPLE WITH A GENTLE TOUCH

There is an easier way to keep things in their place... Find out how easy it can be to attach your Urocare<sup>®</sup> latex leg bag with these fabric straps. Their unique design distributes pressure evenly along the length of the strap offering a vast improvement in comfort over the standard latex leg straps while reducing the risk of bruising, pressure sores, pulled hair, circulation restriction and/or burns caused by rubber straps. The self-gripping fastener can engage anywhere along the length of the strap body making the strap fully adjustable to individual requirements.

Kits are sold by the set, individual straps are sold by the each. These are stocked items.

LATEX-FREE






| Non-Stern | le, Latex-Free<br>FABRIC LEG STRAP KITS                           | FOR UROCARE <sup>®</sup> LATEX LEG BAGS  |                |
|-----------|---|--|----------------|
| Item No.  | Description   | Dimensions <sup>†</sup>                  | Case Item No.  |
| 6351      | X-Large, contains straps ( <sup>#</sup> 6346 & <sup>#</sup> 6345) | See individual straps below              | 635112 (12/CS) |
| 6350      | Large, contains straps ( <sup>#</sup> 6345 & <sup>#</sup> 6344)   | See individual straps below              | 635012 (12/CS) |
| 6349      | Small, contains straps ( <sup>#</sup> 6348 & <sup>#</sup> 6347)   | See individual straps below              | 634912 (12/CS) |
|           | FABRIC LEG STRAP SINGLES  | FOR UROCARE® LATEX LEG BAGS              | 5              |
| Item No.  | Description   | Dimensions <sup>†</sup>                  | Case Item No.  |
| 6348      | Lower Fabric Leg Strap, Small                                     | 7" ~ 13" (18cm ~ 33cm)                   | 634812 (12/CS) |
| 6347      | Upper Fabric Leg Strap, Small                                     | 9" $\sim$ 15" (23cm $\sim$ 38cm)         | 634712 (12/CS) |
| 6346      | Upper Fabric Leg Strap, X-Large                                   | 20" $\sim$ 26" (51cm $\sim$ 66cm)        | 634612 (12/CS) |
| 6345      | Lower Fabric Leg Strap, Large                                     | 12" $\sim$ 17" (31cm $\sim$ 43cm)        | 634512 (12/CS) |
| 6344      | Upper Fabric Leg Strap, Large                                     | 12" $\sim$ 21" (31cm $\sim$ 53cm)        | 634412 (12/CS) |
|           | FABRIC LEG STRAPS FOR U   | ROCARE <sup>®</sup> SPORT LATEX LEG BAGS |                |
| Item No.  | Description   | Dimensions <sup>†</sup>                  | Case Item No.  |
| 6343      | Sport Bag Fabric Strap, Medium $\sim$ Large                       | 19" ~ 30" (48cm ~ 76cm)                  | 634312 (12/CS) |
| 6342      | Sport Bag Fabric Strap, Small ~ Medium                            | 15" ~ 24" (38cm ~ 61cm)                  | 634212 (12/CS) |

Fitz-All<sup>®</sup> Fabric Leg Bag Straps for Uro-Safe<sup>®</sup> and Other Disposable Vinyl Leg Bags

Say Good-Bye to Those Old-Fashioned, Uncomfortable, Sweaty, Rubber Straps!

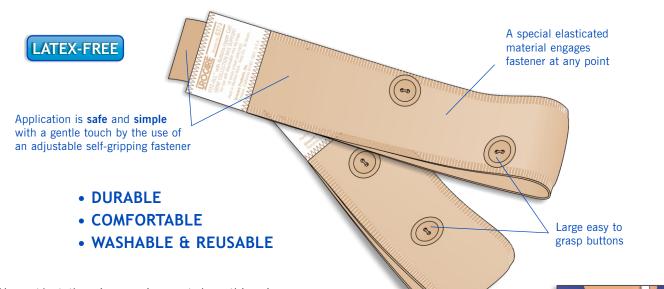

Now, at last, there is an easier way to keep things in their place. Find out how easy it can be to attach a vinyl disposable leg bag with the new Fitz-All<sup>®</sup> urinary leg bag strap.

This universal fabric leg strap will fit most vinyl disposable urinary leg bags and offers a vast improvement in comfort over the standard latex leg straps packaged with most vinyl leg bags.

A special elasticated-loop material distributes pressure evenly along the length of the strap reducing the risk of bruising, pressure sores, pulled hair, circulation restriction and burns caused by ordinary rubber straps.

Large buttons can be easily grasped and slipped through the eyelets of most all vinyl disposable leg bags. The self-gripping fastener can engage anywhere along the length of the strap making the strap fully adjustable to individual requirements. A special blend of elastic and cotton makes them durable and allows them to be hand or machine washed.

#### CONTENTS:

1 Upper Leg Strap (<sup>#</sup>6374) 1 Lower Leg Strap (<sup>#</sup>6375) Illustrated Use and Care Instructions

Sold by the Pair, 1 set per package. This is a stocked item. Non-Sterile, Latex-Free.

| Item No. | Description | Dimensions <sup>†</sup>            |  |  |
|----------|-------------|------------------------------------|--|--|
| 6380     | Universal   | Upper fits up to 24" DIA. (61cm Ø) |  |  |
|          |             | Lower fits up to 18" DIA. (45cm Ø) |  |  |

*†* Approximate dimensions in inches(") and centimeters(cm).

\* Bags must have eyelets for buttons to pass through.






Capacity<sup>‡</sup> Can hold up to a 32 fl. oz. leg bag\* Case Item No. 638012 (12/CS)

*<sup>‡</sup>* Approximate volumes in fluid ounces (fl. oz.) and milliliters (mL).

Fitz-All<sup>®</sup> Fabric Leg Bag Straps for Uro-Safe<sup>®</sup> and Other Disposable Vinyl Leg Bags ELIMINATE BRUISING, PRESSURE SORES, PULLED HAIR, CIRCULATION RESTRICTION AND BURNS CAUSED BY RUBBER STRAPS

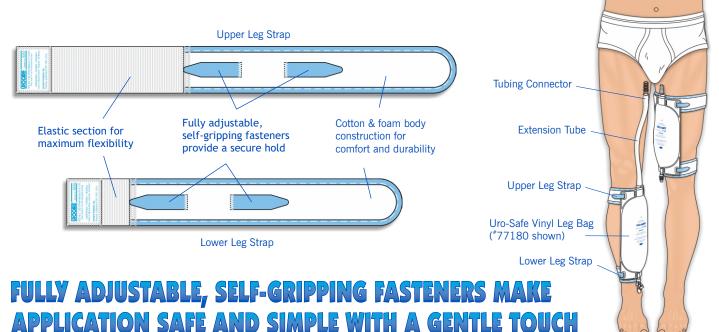

There is an easier way to keep things in their place... Find out how easy it can be to attach your Urocare<sup>®</sup> latex leg bag with these premium fabric leg straps. Their unique design distributes pressure evenly along the length of the strap offering a vast improvement in comfort over the standard latex leg straps while reducing the risk of bruising, pressure sores, pulled hair, circulation restriction and/or burns caused by ordinary rubber straps. The self-gripping fastener can engage anywhere along the length of the strap body making the strap fully adjustable to individual requirements.

Kits are sold by the set, individual straps are sold by the each. These are stocked items. Non-Sterile, Latex-Free.





Case Item No.

637012 (12/CS)

636012 (12/CS)



#### FITZ-ALL® UNIVERSAL FABRIC LEG STRAP "KITS" FOR VINYL LEG BAGS\*

 Item No.
 Description
 Dimensions<sup>†</sup>
 Case

 6370
 Large, contains straps (\*6358 & \*6356)
 See individual straps below
 6370

 6360
 Small, contains straps (\*6357 & \*6355)
 See individual straps below
 6360

|          | FIIZ-ALL UNIVERSAL FA     | ABRIC LEG STRAP SINGLES FOR VINY | L LEG BAGS^    |
|----------|---------------------------|----------------------------------|----------------|
| Item No. | Description               | Dimensions <sup>†</sup>          | Case Item No.  |
| 6358     | Fabric Leg Strap, Large   | 21" ~ 30" (53cm ~ 76cm)          | 635812 (12/CS) |
| 6357     | Fabric Leg Strap, Medium  | 17" ~ 23" (43cm ~ 58cm)          | 635712 (12/CS) |
| 6356     | Fabric Leg Strap, Small   | 13" ~ 20" (33cm ~ 50cm)          | 635612 (12/CS) |
| 6355     | Fabric Leg Strap, X-Small | 9" ~ 13" (23cm ~ 33cm)           | 635512 (12/CS) |

## Straps and Bag Holders

### Uro-Strap<sup>®</sup> Universal Fabric Male External Catheter Strap

### "One Size Fits All"

The Uro-Strap® is an economical solution to numerous changes of tapes and adhesive strips. An elasticized cotton & loop construction allows the self-gripping fastener to attach itself anywhere along the length of the strap making it fully adjustable to individual requirements which provides security and aids in a comfortable fit.


- Easy to Apply & Remove
- Fully Adjustable
- Durable and Reusable
- Hand or Machine Washable


Sold by the each, 1 strap per package. This is a stocked item. Non-Sterile, Latex-Free.

Description Item No 6400 Universal

Dimensions<sup>†</sup> 1" wide by 4.75" Long (2.54cm x 12cm)



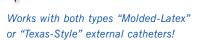

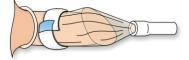

```
Case Item No.
6401 (10/CS)
```







LEARN MORE

LATEX-FREE

### Urocare<sup>®</sup> Universal Fabric Leg Straps

For use with most vinyl disposable or latex reusable leg bags. They are an economical alternative to the standard latex rubber straps packaged with most urine collection bags. An elastic/cotton & body construction allows the self-gripping fastener to attach itself along the length of the strap making it fully adjustable to individual requirements providing a safe, secure hold with a comfortable fit.

- Easy to Apply & Remove
- Fully Adjustable
- Durable and Reusable
- Hand or Machine Washable

Sold by the Pair, 1 set per package. This is a stocked item. Non-Sterile, Latex-Free

Item No. Description 6390 Universal, One size fits all

**Dimensions**<sup>†</sup> Suitable for leg bags up to 44 fl. oz. (1,3 L) or smaller\*

Case Item No. 639012 (12/CS)

*†* Approximate dimensions in inches(") and centimeters(cm).

*‡* Approximate volumes in fluid ounces (fl. oz.) and milliliters (mL)

\* Bags must have eyelets for fasteners to pass through.

Urocare<sup>®</sup> Foley<sup>®</sup> Catheter/Tubing Straps

## ELIMINATE BRUISING, PRESSURE SORES, PULLED HAIR, CIRCULATION RESTRICTION AND BURNS CAUSED BY RUBBER STRAPS

Fully adjustable, self-gripping fastener provides a secure hold

Fastener holds Foley<sup>®</sup> catheters or drainage tubing securely in position for optimum flow Elasticated Loop section for maximum comfort and flexibility

## FULLY ADJUSTABLE, SELF-GRIPPING FASTENERS MAKE APPLICATION SAFE AND SIMPLE WITH A GENTLE TOUCH

There is an easier way to keep things in their place... Find out how easy it can be to attach your Foley<sup>®</sup> catheter or tubing with this elastic fabric strap. It's unique design distributes pressure evenly along the length of the strap offering a vast improvement in comfort while reducing the risk of bruising, pressure sores, pulled hair, circulation restriction and/or burns. The self-gripping fastener can engage anywhere along the length of the strap body making the strap fully adjustable to individual requirements. Hand or machine washable. **Just three(3) easy steps** and your catheter or tubing is held securely in place.

- Easy to Apply & Remove
- Fully Adjustable
- Durable and Reusable
- Hand or Machine Washable

Sold by the each, 1 strap per package. This is a stocked item. Non-Sterile, Latex-Free






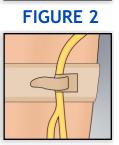


FIGURE 3

| Item No. | Description | Overall Length <sup>†</sup> | Dimensions <sup>†</sup>          |
|----------|-------------|-----------------------------|----------------------------------|
| 6313     | Abdominal   | 34" (86cm)                  | 17" ~ 58" DIA. (43cm ~ 147cm) Ø) |
| 6312     | Large       | 22" (56cm)                  | 11" ~ 37" DIA. (28cm ~ 94cm Ø)   |
| 6311     | Arm         | 14" (36cm)                  | 7" ~ 24" DIA. (18cm ~ 60cm Ø)    |
| 6310     | Standard    | 18" (46cm)                  | 9" ~ 30" DIA. (23cm ~ 78cm Ø)    |



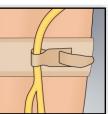

| FIGU          |
|---------------|
| Case Item No. |
| Not Available |
| Not Available |
| Not Available |
| Not Available |

### Straps and Bag Holders

### Urocare<sup>®</sup> Latex and Latex-Free Leg Straps

### Urocare<sup>®</sup> Latex Leg Strap Replacement Set

These straps are replacements for those normally packaged with Urocare<sup>®</sup> Reusable Latex Leg Bags or most other latex or rubber leg bags up to 44 fl. oz. (1,300 mL). Includes white plastic buttons. Straps may also be used with most disposable vinyl leg bags.\*

Sold by the set, 3 straps with buttons per package. This is a stocked item. Non-Sterile, Contains Latex





Item No.Description6006Latex Straps with Buttons

Dimensions<sup>†</sup> 21.75" x 0.75" x 0.06" Capacity<sup>‡</sup> Suitable for leg bags up ( to 44 fl. oz. (1,3L) or smaller

Case Item No. 600610 (10 Sets/CS)

### Urocare<sup>®</sup> "Latex-Free" Leg Straps

These straps are replacement straps for those normally packaged with our Uro-Safe<sup>®</sup> Disposable Vinyl Leg Bags or most other disposable urine collection bags\* up to 32 fl. oz. (1,000 mL). Includes two(2) <sup>#</sup>6034 White Plastic Buttons.

Sold by the pair, 2 straps with buttons per package.

This is a stocked item. Non-Sterile, Latex-Free



Item No.Description60062Latex Straps with Buttons

Dimensions<sup>†</sup> ttons 21.75" x 0.75" x 0.06"





Capacity<sup>†</sup> Suitable for leg bags up to 32 fl. oz. (1L) or smaller

Case Item No. 6006210 (10 Sets/CS)

*†* Approximate dimensions in inches(") and centimeters(cm).

*‡* Approximate volumes in fluid ounces (fl. oz.) and milliliters (mL)

\* Bags must have eyelets for straps to pass through.

LEARN MORE

## Male External Catheters

### Uro-Cath<sup>®</sup> "Molded-Latex" Style Catheters

These male external catheters are commonly referred to as a "Molded-Latex" style because they are dipped from a **single piece** of latex without seams rather than assembled from multiple components.

Each Uro-Cath<sup>®</sup> catheter has a **thin, latex sheath section** similar to a condom at one end—the part that is rolled over the head and shaft of the penis—and a heavier **funnel-shaped, cup section** at the other—from where urine is drained into a bag or other collection device—making them virtually *"twist-proof"* by their unique design.



Uro-Cath<sup>®</sup> catheters range in size from 25mm, our smallest, to 40mm, our largest in 5mm increments to suit a variety of individual requirements. They are packaged individually in a window pouch allowing for quick, easy product identification without opening the pouch. The back side of the pouch contains detailed, illustrated, application instructions relative to the product purchased—no "cookie-cutter" or "one fits all" instructions for all products like some similar products have.

Each catheter may be secured using several different or a combination of methods depending on individual requirements for maximum comfort and security. The product insert included with every box lists a variety of these methods and provides detailed, illustrated, application instructions.

Depending on the item purchased, Uro-Cath<sup>®</sup> male external catheters may also be packaged with adhesive foam strips or protective skin barrier wipes—see chart below for item numbers. For additional information on these products see Urofoam<sup>®</sup> Adhesive Foam Strips or Uro-Prep<sup>®</sup> Protective Skin Barrier Wipes.

Sold: 1 catheter per package/50 packages per case.\* These are stocked items.

|                    | Non-Sterile, Co                                      |  |   |   | US  | Ă                          |
|--------------------|--|--|---|---|-----|----------------------------|
|                    | URO-CATH <sup>®</sup> W                              | ITH UROFOAM  | <sup>®</sup> -2 AND UR                            | 0-PREP <sup>®</sup>   |     |                            |
| CASE               | Item No.<br>5221540*<br>5221535*<br>5221530*         | Description<br>Extra-Large<br>Large<br>Medium          | Size <sup>†</sup><br>40mm<br>35mm<br>30mm         | Case Item No.<br>522154050 (50/CS)<br>522153550 (50/CS)<br>522153050 (50/CS)                  |     | * SOLD ONLY<br>BY THE CASE |
| THE                | URO-CATH <sup>®</sup> W                              | ITH UROFOAM  | <sup>®</sup> -2                                   |   |     |                            |
| ONLY BY            | Item No.<br>522140*<br>522135*<br>522130*            | Description<br>Extra-Large<br>Large<br>Medium          | Size⁺<br>40mm<br>35mm<br>30mm                     | Case Item No.<br>52214050 (50/CS)<br>52213550 (50/CS)<br>52213050 (50/CS)                     |     | * SOLD ONLY<br>BY THE CASE |
| SOLD               | URO-CATH <sup>®</sup> W                              | ITH UROFOAM  | <sup>®</sup> -1                                   |   | V S |                            |
| Y THE CASE *       | Item No.<br>521140*<br>521135*<br>521130*<br>521125* | Description<br>Extra-Large<br>Large<br>Medium<br>Small | Size <sup>†</sup><br>40mm<br>35mm<br>30mm<br>25mm | Case Item No.<br>52114050 (50/CS)<br>52114050 (50/CS)<br>52114050 (50/CS)<br>52114050 (50/CS) |     | * SOLD ONLY<br>BY THE CASE |
| Y B)               |  |  |   |   |     |                            |
| * SOLD ONLY BY THE | Item No.<br>520140*<br>520135*<br>520130*<br>520125* | Description<br>Extra-Large<br>Large<br>Medium<br>Small | Size <sup>†</sup><br>40mm<br>35mm<br>30mm<br>25mm | Case Item No.<br>52014050 (50/CS)<br>52013550 (50/CS)<br>52013050 (50/CS)<br>52012550 (50/CS) |     | * SOLD ONLY<br>BY THE CASE |
|                    |  |  |   |   |     |                            |

\* These items are not sold individually, sold only by the case.

*†* Approximate dimensions in millimeters(mm).

## Male External Catheters

### Uro-Con<sup>®</sup> "Texas-Style" Catheters

These male external catheters are commonly referred to as a "Texas-Style" because they employ a **three-piece construction** technique.

Each Uro-Con<sup>®</sup> catheter has a **thin**, **latex sheath** similar to a condom—which is rolled over the head and shaft of the penis —and is attached by an plastic **catheter insert** to a **4**" **(10cm)** from where urine is drained into a bag or other collection device. Using a "Condom" type natural latex sheath makes these catheters **"fully collapsible"** to prevent pooled urine; a major cause of skin irritation and break-down.



法回

Uro-Con<sup>®</sup> catheters range in size from 25mm, our smallest, to 35mm,

our largest in 5mm increments to suit a variety of individual requirements. They are packaged individually in a window pouch allowing for quick, easy product identification without opening the pouch. The back side of the pouch contains detailed, illustrated, application instructions relative to the product purchased—no "cookie-cutter" or "one fits all" instructions for all products like some similar products have.

Each catheter may be secured using several different or a combination of methods depending on individual requirements for maximum comfort and security. The product insert included with every box lists a variety of these methods and provides detailed, illustrated, application instructions.

Depending on the item purchased, Uro-Cath<sup>®</sup> male external catheters may also be packaged with adhesive foam strips or protective skin barrier wipes—see chart below for item numbers. For additional information on these products see Urofoam<sup>®</sup> Adhesive Foam Strips or Uro-Prep<sup>®</sup> Protective Skin Barrier Wipes.

Sold: 1 catheter per package/50 packages per case.\* These are stocked items. Non-Sterile, Contains Latex.

#### URO-CON<sup>®</sup> WITH UROFOAM<sup>®</sup>-2 AND URO-PREP<sup>®</sup>



\* These items are not sold individually, sold only by the case.

*†* Approximate dimensions in millimeters(mm).

\*

## Male External Catheters

## Uro-Con<sup>®</sup> "Texas-Style" Catheters

These male external catheters are commonly referred to as a "Texas-Style" because they employ a **three-piece construction** technique.

Each Uro-Con<sup>®</sup> catheter has a **thin**, **latex sheath** similar to a condom—which is rolled over the head and shaft of the penis —and is attached by an plastic **catheter insert** to a **2**" (**5cm**) from where urine is drained into a bag or other collection device. Using a "Condom" type natural latex sheath makes these catheters **"fully collapsible"** to prevent pooled urine; a major cause of skin irritation and break-down.

Uro-Con<sup>®</sup> catheters range in size from 25mm, our smallest, to 35mm,

our largest in 5mm increments to suit a variety of individual requirements. They are packaged individually in a window pouch allowing for quick, easy product identification without opening the pouch. The back side of the pouch contains detailed, illustrated, application instructions relative to the product purchased—no "cookie-cutter" or "one fits all" instructions for all products like some similar products have.

Each catheter may be secured using several different or a combination of methods depending on individual requirements for maximum comfort and security. The product insert included with every box lists a variety of these methods and provides detailed, illustrated, application instructions.

Depending on the item purchased, Uro-Cath<sup>®</sup> male external catheters may also be packaged with adhesive foam strips or protective skin barrier wipes—see chart below for item numbers. For additional information on these products see Urofoam<sup>®</sup> Adhesive Foam Strips or Uro-Prep<sup>®</sup> Protective Skin Barrier Wipes.

Sold: 1 catheter per package/50 packages per case.\* These are stocked items. Non-Sterile, Contains Latex.

#### URO-CON® WITH UROFOAM®-2 AND URO-PREP®

\* SOLD ONLY BY THE CASE

SOLD ONLY BY THE CASE

510225\*

| Item No.<br>5122535*<br>5122530*<br>URO-CON® W                       | Description<br>Large<br>Medium                          | Size <sup>†</sup><br>35mm<br>30mm<br>4 <sup>®</sup> -2 | Case Item No.<br>512253550 (50/CS)<br>512253050 (50/CS)                         | * SOLD ONLY<br>BY THE CASE |
|--|---|--|---|----------------------------|
| Item No.<br>512235*<br>512230*                                       | Description<br>Large<br>Medium                          | <sup>Size¹</sup><br>35mm<br>30mm                       | Case Item No.<br>51223550 (50/CS)<br>51223050 (50/CS)                           | * SOLD ONLY<br>BY THE CASE |
| URO-CON <sup>®</sup> W<br>Item No.*<br>511235*<br>511230*<br>511225* | VITH UROFOAN<br>Description<br>Large<br>Medium<br>Small | <mark>8<sup>™</sup>-1</mark><br>35mm<br>30mm<br>25mm   | Case Item No.<br>51123550 (50/CS)<br>51123050 (50/CS)<br>51122550 (50/CS)       | * SOLD ONLY<br>BY THE CASE |
| URO-CON <sup>®</sup> M<br>Item No.<br>510235*<br>510230*<br>510225*  | Description<br>Large<br>Medium                          | Size <sup>†</sup><br>35mm<br>30mm                      | ER<br>Case Item No.<br>51023550 (50/CS)<br>51023550 (50/CS)<br>51023550 (50/CS) | * SOLD ONLY<br>BY THE CASE |

74

51023550 (50/CS)

\* These items are not sold individually, sold only by the case.

25mm

*†* Approximate dimensions in millimeters(mm).

Product Catalog 2018 • Urocare Products, Inc.

Small








## Male External Catheters

### Uro-Con<sup>®</sup> "Texas-Style" Catheter Components to BUILD YOUR OWN

These male external catheters are commonly referred to as a "Texas-Style" because they employ a three-piece construction technique.

Each Uro-Con<sup>®</sup> catheter has a thin, latex sheath similar to a condom—which is rolled over the head and shaft of the penis —and is attached by an plastic catheter insert to a 2" (5cm) or 4" (10cm) from where urine is drained into a bag or other collection device. Using a "Condom" type natural latex sheath makes these catheters "fully collapsible" to prevent pooled urine; a major cause of skin irritation and break-down.

If you would like to build your own male external catheter to get just the right fit, size or a custom tube length then you are in the right place. Below are all the components, which are sold separately, that are used to make-up the Uro-Con<sup>®</sup> male external catheters: condom sheaths, inserts and tubing.

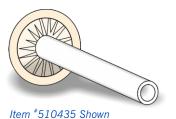



### Urocare<sup>®</sup> "High-Quality" Latex Sheaths

These latex sheaths are used in conjunction with Uro-Con<sup>®</sup> Catheter Inserts and pliable white-silicone tubing to construct Uro-Con<sup>®</sup> "Texas-Style" male external catheters—refer to the other components on this page for more information.



### **Uro-Con<sup>®</sup>** Catheter Inserts

Used to join Uro-Con<sup>®</sup> "High-Quality" Latex Sheaths and pliable, white-silicone tubing to make Uro-Con<sup>®</sup> "Texas-Style" male external catheters—see other components on this page.

Sold by the box, 100 inserts per box. This is a stocked item. Non-Sterile, Latex-Free



Item No. Description 6007 Uro-Con<sup>®</sup> Catheter Inserts

Dimensions 0.35" O.D. x 0.5" (9mm Ø x 1.2cm)



Case Item No. Not Available

Urocare<sup>®</sup> White-Silicone Elastomer Drainage Tubing This tubing may be easily cut to desired length and used as an extension between Uro-Con<sup>®</sup> or Uro-Cath<sup>®</sup> male external catheters, Foley<sup>®</sup> indwelling catheters and reusable or disposable urinary drainage bags, bottles or other urinary and ostomy

LATEX-FREE

appliances. Provides a snug fit on most standard sized drainage ports/outlets while resisting kinks.

Sold by the each, 1 tube per package. This is a stocked item. Non-Sterile, Latex-Free

Item No. Description **Dimensions**<sup>†</sup> 600831 0.31" I.D. x 10 ft. (8mm Ø x 3m) long Large



Case Item No. Not Available



## Enuresis (Bed-Wetting) Treatment Systems

with break-through technologies!

## THE MOST EFFECTIVE METHOD TO TREAT BED-WETTING, IS TO USE A BED-WETTING ALARM

Does your child suffer from problem bed-wetting? Bed-wetting affects millions of people worldwide and is no fault of the child. It can be treated and cured. For **over 30 years** the creators of the DRI Excel<sup>®</sup> and DRI Eclipse<sup>®</sup> enuresis treatment alarms have cured bed-wetting problems all around the world. These alarms continue to offer the **most effective & proven** bed-wetting treatment on the market. Below is more information on why bed-wetting happens and how the alarms effectively treat bed-wetting.

- It's **not usual** to consider bed-wetting a problem until after the age of 5.
- Most **bed-wetting is genetic** it runs in the family.
- A typical bed-wetting pattern is **heavy sleeping and producing too much urine**.
- Cutting down on fluids at night makes no difference.
- The **most effective method** is to use an alarm that's triggered when wetting starts.

#### Here's how the DRI Eclipse<sup>®</sup> and DRI Excel<sup>®</sup> bed-wetting alarms work:



The DRI Eclipse<sup>®</sup> wireless alarm unit is placed on a table, or the DRI Excel<sup>®</sup> alarm unit goes on the shoulder. The Urosensor<sup>™</sup> goes in the underwear. When the Urosensor<sup>™</sup> is wet with urine, it triggers a loud alarm to wake the bed-wetter. Over the next few days or weeks, your child begins to learn to wake more and more quickly, until he/she is beating the alarm and waking before wetting starts. The pattern now becomes waking up every time he or she needs to go to the bathroom. Then the final stage begins; your child will sleep through the night without wetting the bed or needing to wake up to go to the bathroom. This is the mature pattern, your child is capable of waking if she or he needs to go to the bathroom and even more interesting, it now appears that the overproduction of urine has stopped.

#### DRI Eclipse<sup>®</sup> and DRI Excel<sup>®</sup> Bed-wetting Alarm Testimonials:

Don't just take our word for it, read what parents of bed-wetters who have used the alarms have to say.

"He is nine years old and has always had a bed-wetting problem... we never liked giving him medicine, so we ordered the DRI Excel<sup>®</sup>. It only took a few weeks for our son to stop wetting the bed. We were afraid to say he was cured, so we waited, and now three months later he's still doing great! We never tried a bed-wetting alarm because we were skeptical of them, now we say, "we should have tried one years ago." The Johnsons

"I never thought it would happen we struggled so long with our sons toileting habits but after only about 1 month of using the sleep alarm he now stays dry every night! He has not even had any little accidents. Your product works great and is so affordable. We have neighbors that spent over \$1,500 for a program that does the same thing yours does—most of that was for support—and you provided that FREE. Thank you for such a great product!"

The Askins, MI, USA

"The DRI Excel<sup>®</sup> has worked wonders. Thank you for your advice and for taking time to get in touch, it was very much appreciated." T. Cliff - Australia

**Read the online scientific report on bed-wetting at** www.urocare.com/EN/Products/EnuresisAlarmsResearchFAQEN.php. This report can answer your questions and provide you with assurance that DRI Eclipse<sup>®</sup> and DRI Excel<sup>®</sup> alarms do work.

Sold by the each, 1 alarm with Urosensor<sup>™</sup> & batteries per box. This is a stocked item. Non-Sterile, Latex-Free





| Item No. | Description   |
|----------|---|
| 4610     | DRI Eclipse <sup>®</sup> " <b>wireless"</b> enuresis alarm <sup>‡</sup> |
| 4600     | DRI Excel <sup>®</sup> "corded" enuresis alarm                          |

Dimensions<sup>†</sup> 3" x 1.5" x 0.75" (8cm x 4cm x 2cm) 2.25" x 1.5" x 0.75" (6cm x 4cm x 2cm) Case Item No. Not Available Not Available

*†* Approximate dimensions in inches(") and centimeters(cm).

*‡* Distance depends somewhat on the situation and number of obstacles. Actual distance may vary.

26

## Enuresis (Bed-Wetting) Treatment Systems

### Replacement Components/Parts

The DRI Excel<sup>®</sup> and DRI Eclipse<sup>®</sup> enuresis treatment alarms don't have any replaceable parts other than the urine sensors, called the Urosensor<sup>™</sup>, which are a major component of the alarm's success. What appears to be a very simple part of the product is in fact a complex design challenge. The sensor has a tough job – every night it's wet by urine and then washed over and over again; it gets tugged and pulled during sleep ("corded model") and it has the corrosive factor of urine to contend with.

> IMPORTANT NOTE: The sensor is the key to beating bed-wetting. Don't be fooled by tiny clips or domes on the underwear, the urine can miss (especially with boys) and not trigger the alarm!

### Urosensor<sup>™</sup> for DRI Excel<sup>®</sup> (Plug-In or "Corded" model)

Replacement remote sensor for DRI Excel<sup>®</sup> (<sup>#</sup>4600). Over the past **25 years** the creators have designed and evaluated the urine sensor again and again. The result is the Urosensor<sup>™</sup>: a **unique**, patented, **non-metallic**, completely molded, flexible product. It cannot corrode, irritate the skin or allow urine to leak inside and break. Because the urine cannot leak inside, it can be washed guickly to be instantly reused; it does not get contaminated with the urine, and internal wiring to the alarm cannot become corroded. The Urosensor<sup>™</sup> on the DRI Excel<sup>®</sup> (plug-in or "corded" model) is a

consumable item and it is expected that it will need replacing from time to time depending upon its duration of use, just as earphones on a portable radio will need replacing from time to time. Remember that earphones may last varying times depending on how carefully or otherwise they are used, especially when being used by a child and so the life of the Urosensor<sup>™</sup> will vary. It cannot be predicted exactly how long the sensor will last-for this reason our strong advice is to purchase a spare Urosensor<sup>™</sup> when you buy your DRI Excel<sup>®</sup> so that there is no break in treatment while waiting for a replacement to arrive.

Sold by the each, 1 sensor per package. This is a stocked item. Non-Sterile, Latex-Free.

Item No. Description Urosensor<sup>™</sup> (plug-in type) 4603

LATEX-FREE

**Dimensions**<sup>†</sup>

Sensor Pad: 0.25" x 1.63" (0.64cm x 2cm x 4.3cm)





Color may vary





Case Item No.

Not Available

Urosensor<sup>™</sup> for DRI Eclipse<sup>®</sup> ("Wireless" model)

Replacement remote sensor for DRI Excel<sup>®</sup> (<sup>#</sup>4610). The "wireless" Urosensor™ Remote presents an even greater challenge than the plug-in model. To be wireless, it has to communicate with the alarm by a radio signal. Therefore it has electronics and a battery inside. These have to be miniature to keep the sensor small and comfortable. It must be completely sealed to keep the urine out and the battery small yet durable enough to last for a long time not just 7 days or 1 month as in some other alarms.

The Urosensor<sup>™</sup> Remote is the only 100% wireless sensor to have solved all of these complex issues which enable the sensor to be small, completely sealed, located right in the crotch of the underwear where it is triggered immediately by the urine and for the battery to last for at least three (3) years.

Sold by the each, 1 sensor per package. This is a stocked item. Non-Sterile, Latex-Free.

Item No. Description 4613 Urosensor<sup>™</sup> Remote "wireless"

ATEX-FREE

Cord: 35" (89cm)

**Dimensions**<sup>†</sup> 0.38" x 1" x 2" (1cm x 2.5cm x 5cm)





Case Item No. Not Available

*†* Approximate dimensions in inches(") and centimeters(cm).

‡ Distance depends somewhat on the situation and number of obstacles. Actual distance may vary.



## Adhesives & Removers

### Uro-Bond<sup>®</sup> Brush-On Silicone Adhesives - 100% Latex Free!

#### **Completely Transparent and Highly Effective!**

Uro-Bond<sup>®</sup> brush-on silicone adhesives are pressure-sensitive silicone adhesives specifically formulated for unique bonding applications and are suitable for adhering non-permanent material to the body surface.

Features include the retention of adhesive gualities in the presence of moisture or perspiration, their adhesive properties are unaffected by normal temperature variations or time and they adhere to many surfaces such as skin, metals, glass, paper, fabric, plastics, silicone, organic rubber products, etc.

Possible applications for Uro-Bond<sup>®</sup> adhesives might include, but are not limited to, adhering dressings, prosthetics and metal or plastic devices to the body, securing male external urinary catheters, adhering colostomy, ileostomy and tracheostomy devices, attaching electrodes to the skin for patient monitoring, affixing surgical dressings and pads to the skin and adhering maxillo-facial prosthetic devices.

#### **KEY FEATURES:**

- Silicone based—for best biocompatibility.
- Completely Transparant—Excellent for facial prosthetics, tracheostomys and toupées.
- Adheres to many surfaces—including other silicones.
- Water and perspiration resistant—It retains it's adhesive qualities even under water.
- Brush adhesive just where you need it—No messy sprays which get adhesive everywhere.

#### Securing Male External Catheters

The most effective way to secure a male external catheter is to use Uro-Bond® brush-on silicone adhesive. With Uro-Bond<sup>®</sup> a male external catheter can be held in place for up to three days-results may vary, but it's nice to know that when you need to depend on an adhesive, you can depend on Uro-Bond<sup>®</sup>.

#### Securing Prosthetics, Tracheostomy Devices, Toupées etc.

Application of prosthetics and devices to the skin can be tricky and may

require some practice before the exact method that works best for you is determined. It is recommended that Uro-Bond<sup>®</sup> be applied to both surfaces and allowed to become tacky before adhering the surfaces together. This will ensure the maximum hold.

#### **Uro-Bond**<sup>®</sup> Thinner

The use of Uro-Bond<sup>®</sup> Thinner may be deemed necessary where Uro-Bond<sup>®</sup> Brush-on Silicone Adhesives have thickened—when the bottle is opened frequently, solvent evaporation can occur.

Suggested uses include, but are not limited to, use as a dilutant for Uro-Bond<sup>®</sup> Brush-on Silicone Adhesive as well as other medical adhesives, as a degreasing agent or a solvent. However, Urocare<sup>®</sup> does not recommend the use of Uro-Bond<sup>®</sup> Thinner as an adhesive remover—Urocare<sup>®</sup> recommends using Urocare<sup>®</sup> Adhesive Remover Pads (<sup>#</sup>5600) for that purpose.

Sold by the bottle, 1 bottle per box. These are stocked items. Non-Sterile, Latex-Free.

*†* Approximate volumes in fluid ounces (fl. oz.) and milliliters (mL)



#### Flammablility Flammable Flammable Non-Flammable Non-Flammable Flammable Flammable

Case Item No. 50150312 (12/CS) 50100312 (12/CS) 50050312 (12/CS) 500501512 (12/CS) 50000312 (12/CS) 500001512 (12/CS)

### Quality Health Care Products Since 197

Uro-Bond<sup>®</sup> III

Brush-On

Silicone

Adhesive

Original

Formula

Brush-On

Silicone

Adhesive





LATEX-FREE

Size<sup>†</sup>

3 fl. oz. (88.7 mL)

3 fl. oz. (88.7 mL)

3 fl. oz. (88.7 mL)

1.5 fl. oz. (44.4 mL)







URO-BOND® III

## Adhesives & Removers

### Tracho-Foam<sup>®</sup> Adhesive Foam Tracheostomy Discs/Wafers

#### "Conforms easily to the irregular contours of the body"

Tracho-Foam<sup>®</sup> adhesive foam discs are soft, pliable foam discs specifically designed for the unique bonding applications of appliances to the skin and are suitable for securing valve housings of voice restoration systems to the body.

With adhesive coated on both sides, they provide a comfortable, air-tight seal that conforms to the irregular contours of the stoma and eliminate hissing-a side effect often found in other thinner or less flexible products.

#### **KEY FEATURES:**



- Ensolite Foam—Hypo-Allergenic.
- Closed-Cell Foam Structure—for an air-tight seal that eliminates hissing.
- **Soft and Pliable**—*Easily conforms to irregular contours of the body.*
- Perspiration Resistant—It retains it's adhesive qualities even when sweating.
- Easy to Apply and Remove—Just peel 'n' Stick to apply, peel back to remove.

If the disc is difficult to remove, Urocare<sup>®</sup> Adhesive Remover Pads (<sup>#</sup>5600) may be used to soften the adhesive and assist in the removal of the disc. They contain a non-toxic, citrus-based adhesive remover that may also be used to remove most types of medical adhesive or adhesive residue from skin and appliances or in preparation for use of a new disc. Most medical grade adhesive removers will work basically in the same manner; however, consult your doctor or speech therapist to determine the best product for you and how to use it safely. For more information see Urocare<sup>®</sup> Adhesive Remover Pads.

In the event of skin that has become sensitive to aggressive medical adhesives, Urocare<sup>®</sup> has developed Uro-Prep<sup>®</sup> Protective Skin Barrier Wipes (#5500) which may be used under all types of medical adhesives. They protect delicate, sensitive skin from aggressive adhesives often used in conjunction with urinary or ostomy appliances. Uro-Prep<sup>®</sup> wipes are safe, effective, and for external use only. For more information see Uro-Prep® Protective Barrier Wipes.


Sold by the box, 30 discs/wafers per box. These are stocked items. Non-Sterile, Latex-Free.

ATEX-FREE

Item No. Description Size 0.13" x 1" ID x 2.63" OD (0.3cm x 2.5cm Ø x 6.7cm Ø) 5310 Large 0.13" x 1" ID x 2.25" OD (0.3cm x 2.5cm Ø x 5.7cm Ø) 5300 Standard

*†* Approximate dimensions in inches(") and centimeters(cm).



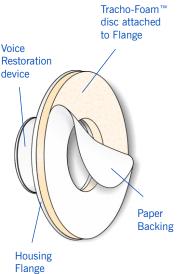



Case Item No. Not Available Not Available



5300

Item #5300 Shown

### Adhesives & Removers

### Urofoam<sup>®</sup> Adhesive Foam Strips

#### "Conforms easily to the irregular contours of the body"

Urofoam<sup>®</sup> double or single-sided adhesive foam strips are soft, pliable foam strips specifically designed for the unique bonding applications of male external catheters to the skin and are suitable for use with both standard and self- adhering type catheters.

They provide a comfortable, air-tight seal that is an effective way to secure a male external catheter to the body. They are easy to apply and remove so when it comes time to change that catheter, it can be changed easily and quickly.

It is recommended that a male external catheter be changed once a day to promote healthy skin, however, with the use or Urofoam<sup>®</sup> strips, a male external catheter can be held in place for up to three days—results may vary, but it's nice to know that when you need to depend on an adhesive strip, you can depend on Urofoam<sup>®</sup>.

#### **KEY FEATURES:**

- Ensolite Foam—Hypo-Allergenic.
- Closed-Cell Foam Structure—for an air-tight seal.
- Soft and Pliable—Easily conforms to irregular contours of the body.
- Perspiration Resistant—It retains it's adhesive qualities even when sweating.
- Easy to Apply and Remove—Just peel 'n' Stick to apply, peel back to remove.

If the strip is difficult to remove, Urocare<sup>®</sup> Adhesive Remover Pads (<sup>#</sup>5600) may be used to soften the adhesive and assist in the removal of the strip. They contain a **non-toxic**, **citrus-based** adhesive remover that may also be used to remove most types of medical adhesive or adhesive residue from skin and appliances or in preparation for use of a new strip.

Most medical grade adhesive removers will work basically in the same manner; however, consult your doctor to decide the best product for you and how to use it safely. For more information—see Urocare<sup>®</sup> Adhesive Remover Pads.

In the event of skin that has become sensitive to aggressive medical adhesives, Urocare<sup>®</sup> has developed Uro-Prep<sup>®</sup> Protective Skin Barrier Wipes (<sup>#</sup>5500) which may be used under all types of medical adhesives. They protect delicate, sensitive skin from aggressive adhesives often used in conjunction with urinary or ostomy appliances. Uro-Prep<sup>®</sup> wipes are safe, effective, and for external use only. For more information see Uro-Prep<sup>®</sup> Protective Barrier Wipes.

Item No.Description5200Double-Sided5100Single-Sided

LATEX-FREE



Size<sup>†</sup> 0.09" x 1" x 5.75" (0.3cm x 2.5cm x 14.6cm) 0.09" x 0.88" x 5.75" (0.3cm x 2.2cm x 14.6cm)





FIGURE A: Urofoam<sup>®</sup>-2 Method (Foam strip under catheter)



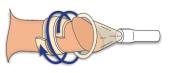




FIGURE B: Urofoam<sup>®</sup>-1 Method (Foam strip over catheter)

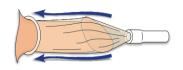

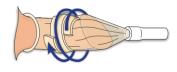

Case Item No. Not Available Not Available

Sold by the box, 50 strips per box. These are stocked items. Non-Sterile, Latex-Free.

## Adhesives & Removers

### Urocare<sup>®</sup> Adhesive Remover Pads

Urocare<sup>®</sup> adhesive remover has been specifically formulated for dissolving and removing unwanted adhesive or adhesive residue from urinary and ostomy appliances, tens units, monitoring electrodes, skin and can even be used for the removal of tar.

Packaged in a small, convenient wipe, Urocare<sup>®</sup> adhesive remover pads are the perfect solution to aid in the removal of adhesive residue left behind by male external catheters.

#### **KEY FEATURES:**

- Soap & Water Clean-Up—Just wash, rinse and dry.
- Citrus-Based—Citrus scent and biodegradable.
- Many Uses—Use on nitro patches, tens units, electrodes, even tar.
- Convenient—Travel size pads are small and easy to use.
- NON-TOXIC—Lab Tested, non-toxic and free from harsh solvents.

They may be used to soften the adhesive of double-sided adhesive foam strips as well as most brush-on medical adhesives. They contain a lab tested, non-toxic, citrus-based solution that may also be used to remove these types of medical adhesives or adhesive residue from skin and appliances in preparation for the use of a new adhesive strip/wafer.

It is recommended that a male external catheter be changed once a day to promote healthy skin. With the use of Urocare<sup>®</sup> adhesive remover pads, a male external catheter can be changed quickly and easily.

- Suitable for Hospital, Long-Term Care and Home Use.
- Alcohol-Free Formula.
- Environmentally Friendly—Does not contain ozone-depleting chlorofluorocarbons.
- Convenient Packaging—Use only what you need so there's less waste!



CIT



Sold by the box, 50 individual wipes/pads per box. This is a stocked item. Non-Sterile, Latex-Free. Note: For external use only.

**ATEX-FREE** 

MADE IN

USA

#### **ACTUAL SIZE FRONT & BACK**

Item No. Description Size 5600 Adhesive Remover Wipe 0.09" x 2" x 2.25" (0.3cm x 5cm x 5.7cm)

Case Item No. Not Available

IIRO-PDI

### Skin Care Products

### Uro-Prep<sup>®</sup> Protective Skin Barrier Wipes/Pads

Uro-Prep<sup>®</sup> Protective Barrier Wipes have been specifically formulated to create a protective barrier between delicate, sensitive skin and aggressive adhesives. They can be used under all types of medical adhesives.

Packaged in a small, convenient wipe, Uro-Prep<sup>®</sup> skin barrier wipes are the perfect solution to aid in the protection of delicate skin that has become sensitive to repeated use of medical adhesives; especially, those used to secure male external urinary catheters and urinary or ostomy appliances.

Uro-Prep<sup>®</sup> protective skin barrier wipes are alcohol based which are easy to apply, safe, effective and for external use only.

#### **KEY FEATURES:**

- Alcohol Based—Dries quickly for fast use.
- Many Uses—Can be used under all types of medical adhesives.
- Convenient—Travel size pads are small and easy to use.
- Soap & Water Clean-Up—Just wash, rinse and dry.

It is recommended that a male external catheter be changed once a day to promote healthy skin and using Uro-Prep<sup>®</sup> protective skin barrier wipes before applying a male external catheter, urinary or ostomy appliance will help protect skin from any aggressive adhesives used with their application.

- Suitable for Hospital, Long-Term Care and Home Use.
- Alcohol-Free Formula.
- Environmentally Friendly—Does not contain ozone-depleting chlorofluorocarbons.
- Convenient Packaging—Use only what you need so there's less waste!



| LATEX-FREE  | ROTECTIVE SKIN BARRIER WIPES<br>POTECTIVE SKIN BARRIER WIPES<br>POR URINARY<br>POR URINARY<br>Softwares<br>Constant of the stand<br>Stand Stormy Appliances<br>Use under Adhesives, Male External<br>Use Use Use Use Use Use Use Use Use Use | URO-PREP# 5000Therefores: Wash area with mild some with sessing application and allow to dry. Protective sessing applicator and allow to dry. Protective applicator and allow to dry.Matter applicator and allow to dry. Protective applicator applic |
|---|--|---|
| Sold by the box, 50 individual wipes/pads per This is a stocked item. | er box.<br>ACTUAL SIZE FR  | ONT & BACK  |

This is a stocked item. Non-Sterile, Latex-Free, Contains Alcohol. **Note:** For external use only.

Item No.DescriptionSize1()5500Protective Barrier Wipes0.09" x 2" x 2.25" (0.3cm x 5cm x 5.7cm)N

Case Item No. Not Available

### Cleansers

### Urolux<sup>®</sup> Urinary and Ostomy Appliance Cleanser & Deodorant

Rinsing alone does not clean a urinary or ostomy appliance and the use of vinegar, bleaches or toilet bowl cleaners only hardens, distorts and damages rubber, latex and plastic parts-not to mention the unpleasant and embarrassing odors often created by the use of these chemicals.

## DON'T LET THIS HAPPEN TO YOUR URINARY APPLIANCE!



Urocare<sup>®</sup> recommends the use of Urolux<sup>®</sup> urinary and ostomy appliance cleanser and deodorant for cleaning latex or rubber urinary leg bags, male urinal sheaths, urinary drainage bottles, urinary drainage and extension tubing or any other reusable urinary or ostomy appliance.

Urolux<sup>®</sup> is a concentrated solution specifically formulated to clean and deodorize rubber, latex or plastic urinary and ostomy appliances. Crystal build-ups are dissolved and unpleasant odors are eliminated.

#### **KEY FEATURES:**

- Effectively Cleans the Entire Fluid Pathway
- Fresh Citrus Scent
- Concentrated Formula Goes A Long Way
- Dissolves Calcium and Urine Crystal Build-Ups
- For use with Urinary & Ostomy Appliances

Sold by the bottle.


This is a stocked item. Non-Sterile, Latex-Free, Contains Phosphoric Acid. Note: For external use only. May be a skin and eye irritant.

Description Item No. 700216 Urolux<sup>®</sup> Cleanser, Standard 700204 Urolux<sup>®</sup> Cleanser, Trial Size

Size<sup>†</sup> 16 fl. oz. (473 mL) 4 fl. oz. (118 mL)

Case Item No. 70021612 (12/CS) Not Available

4 fl. oz. (118 ml.

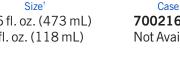

IADE IN

RESH

VJJS

### Urocare<sup>®</sup> Plastic/Small Parts Cleaning Brush

This brush is made with durable, white-nylon bristles and a malleable, twisted, stainless-steel wire body. It may be shaped to fit almost any contour and is especially good for removing uric crystal build-ups from within urinary and ostomy appliances. May be used with any urinary or ostomy appliance that has small parts, tight corners, small holes and other hard to reach areas that need periodic brushing to loosen calcium and uric crystal build-ups.

| Sold by the each, 1 brush per package.<br>This is a stocked item.<br>Non-Sterile, Latex-Free |                               |  | 0   |         |                                 |  |
|--|-------------------------------|--|---|---------|---------------------------------|--|
| Item No.<br>7003   | Description<br>Cleaning Brush |  | Dimensions <sup>‡</sup><br>D.D. x 8" (1.9cm Ø x | ( 20cm) | Case Item No.<br>700324 (24/CS) |  |





Reorder No. 700216

INCENTRATED FORMULA

Directions, Precautions &

Antidote Instructions on Back Panel

ER; Harmful or fatal if swallowed. NLY AS DIRECTED. Read all precau Igs on back panel carefully befor OUT OF REACH OF CHILDREN

16 fl. oz. (473 ml.)

Db

LEARN MORE

For Use with Most Any Urinary or Ostomy Appliance Urocare<sup>®</sup> Clear-Vinyl Drainage & Extension Tubing May be used as an extension between male external catheters, Foley<sup>®</sup> indwelling catheters and reusable or disposable urinary drainage bags, bottles or other urinary and ostomy appliances. Provides a snug fit on most standard sized drainage ports/outlets while resisting kinks. May be easily cut to desired length. \*6085 #6086 #6087 #6020 \*6060 #6061 60083<sup>-</sup> IIIIII w IIIIII m lillill m #6019 #6018 #600831 #6016 Sold by the each, 1 tube per package. LEARN MORE #6017 This is a stocked item. See list below for item sterility. LATEX-FREE **DEHP-FREE** MADE IN All are DEHP & Latex-Free. JSA **CLEAR-VINYL DRAINAGE AND EXTENSION TUBING** Item No. **Dimensions**<sup>†</sup> Sterile Case Item No. Description 6087 Tube with Adaptor & Cap 0.28" I.D. x 18" (7mm Ø x 45cm) long Sterile (EO) 608712 (12/CS) Tube with Adaptor & Cap 0.28" I.D. x 18" (7mm Ø x 45cm) long 6086 Non-Sterile 608612 (12/CS) 6085 Tube with Adaptor & Cap 0.28" I.D. x 8.5" (7mm Ø x 21.6cm) long Non-Sterile Not Available 6061 Tube with Adaptor & Cap 0.28" I.D. x 5' (7mm Ø x 1.5m) long Non-Sterile 606150 (50/CS) 6060 Tube with Adaptor & Cap 0.28" I.D. x 5' (7mm Ø x 1.5m) long 606050 (50/CS) Sterile (EO) Tube with Connector (#6010) 6020 0.31" I.D. x 18" (8mm Ø x 45cm) long Sterile (EO) 602012 (12/CS) 6019 Tube with Connector ( $^{#}6010$ ) 0.31" I.D. x 18" (8mm Ø x 45cm) long Non-Sterile 601912 (12/CS) WHITE-RUBBER/SILICONE ELASTOMER DRAINAGE AND EXTENSION TUBING Dimensions<sup>1</sup> Sterile Item No. Description Case Item No. 6018 Tube with Connector (#6010) 0.31" I.D. x 18" (8mm Ø x 45cm) long Non-Sterile 601812 (12/CS) & Clamp (#7000)

|        | & Glamp ( 7000)             |                                       |              |                |  |
|--------|-----------------------------|---------------------------------------|--------------|----------------|--|
| 6017   | Tube with Connector (#6010) | 0.31" I.D. x 18" (8mm Ø x 45cm) long  | Sterile (EO) | 601712 (12/CS) |  |
| 6016   | Tube with Connector (#6010) | 0.31" I.D. x 18" (8mm Ø x 45cm) long  | Non-Sterile  | 601612 (12/CS) |  |
| 600831 | Tubing, Large               | 0.31" I.D. x 10 ft. (8mm Ø x 3m) long | Non-Sterile  | Not Available  |  |
| 600828 | Tubing, Medium              | 0.28" I.D. x 10 ft. (7mm Ø x 3m) long | Non-Sterile  | Not Available  |  |
| 600825 | Tubing, Small               | 0.25" I.D. x 10 ft. (6mm Ø x 3m) long | Non-Sterile  | Not Available  |  |
|        |                             |                                       |              |                |  |

*†* Approximate dimensions in inches(in) and centimeters(cm).

Drainage and Extension Tubing

Quality Health Care Products Since 197

## Tubing Connectors and Clamps

### For Use with Most Any Urinary or Ostomy Appliance

### Urocare<sup>®</sup> Two-Position Tube Clamp

Replacement thumb-clamp found on the Urocare<sup>®</sup> Straight-Thru Adaptor with Thumb Clamp (<sup>#</sup>6005) and white-rubber or /silicone elastomer tubing (<sup>#</sup>6018). Provides an easy on/off flow control/closure. It allows either a free-flow (on) or complete restriction (off) of fluids passing through latex, silicone or white-rubber tubing 0.38" O.D. (0.95cm Ø) or less. It is not recommended for clear-vinyl tubing as this type of tubing tends to be more rigid and may not hold.

Sold by the box, 10 clamps per box. This is a stocked item. Non-Sterile, DEHP & Latex-Free.

LATEX-FREE





Item No. Description 700010 Two-Position Thumb Clamp, White 1.63" x 0.75" x 0.81" (4.1cm x 1.9cm x 2cm)

Dimensions<sup>1</sup>



### Urocare<sup>®</sup> Adjustable Tube Clamp

Provides an incremental, adjustable closure allowing a reduction or complete restriction of fluids passing through clear-vinyl, white-rubber, silicone elastomer or latex tubing 0.44" O.D. (1.1cm Ø) or less. Replacement thumb-clamp found in the Urocare<sup>®</sup> Urinary Drainage Bottle (<sup>#</sup>4100). LEARN MORE

Sold by the box, 10 tube clamps per box. This is a stocked item. Non-Sterile, DEHP & Latex-Free.







Case Item No. 699950 (50/CS)

IISA

Item No. Description Adjustable Thumb Clamp, White 699910

Dimensions 1.5" x 1" x 0.7" (3.8cm x 2.54cm x 1.7cm)

## Urocare<sup>®</sup> Catheter Connectors

Features "Reverse-Grip" technology and graduated barbs to provide a secure hold on most standard sized drainage or extension tubing. May be used to connect male external catheters, Foley<sup>®</sup> indwelling catheters and reusable or disposable urinary drainage bags, bottles or other urinary and ostomy appliances.

Sold by the box, 10 connectors per box. MADE IN This is a stocked item. LATEX-FREE USΔ Item #6014 Shown Non-Sterile, DEHP & Latex-Free, Item No. Description **Dimensions**<sup>†</sup> 6014 Catheter Connector, Large 0.38" O.D. x 3" long (9mm Ø x 8cm)





6013 Catheter Connector, Small

### Urocare<sup>®</sup> Tubing Connectors

Replacement tubing connector found preassembled on Urocare<sup>®</sup> drainage & extension tubing, Urocare<sup>®</sup> catheter connectors and on tubing included in Uro-Kit<sup>®</sup> Reusable Latex Urinary Leg Bag Systems. Features "Reverse-Grip" technology and graduated barbs to provide a secure hold on most standard sized drainage or extension tubing.

0.31" O.D. x 3" long (8mm Ø x 8cm)

May be used to connect male external catheters, Foley<sup>®</sup> indwelling catheters and reusable or disposable urinary drainage bags, bottles or other urinary and ostomy appliances.

**Dimensions**<sup>†</sup>

0.38" O.D. x 2.25" long (9mm Ø x 6cm)

0.31" O.D. x 2.25" long (8mm Ø x 6cm)

| Sold by the box, 10 connectors per box. |  |  |  |  |
|---|--|--|--|--|
| This is a stocked item.                 |  |  |  |  |
| Non-Sterile, DEHP & Latex-Free.         |  |  |  |  |

Item No. Description 601010 Tubing Connector, Large 600910 Tubing Connector, Small

*†* Approximate dimensions in inches(in) and centimeters(cm).





Case Item No. 601050 (50/CS) 600950 (50/CS)

Closed

Open

## Adaptors and Drain Valves

### Urocare<sup>®</sup> Quick Drain Valve<sup>™</sup>

Only a limited amount of finger or hand control is required to operate these valves independently. Finger loops or a crossbar may be attached to the lever to suit individual requirements—see *Figure A*. Fits Urocare<sup>®</sup> Reusable Latex Leg Bags and Urocare<sup>®</sup> Male Urinal Sheaths only.

Made with pliable, white-rubber tubing and plastic—**no metal parts to corrode**. May be used as a replacement for the Standard Bottom Drain Valve (#6004) preassembled on all Urocare<sup>®</sup> Reusable Latex Leg Bags and Urocare<sup>®</sup> Male Urinal Sheaths.

#### **KEY FEATURES:**

- · Lever operated for quick, controlled drainage
- Threads are sealed to prevent urine from touching the skin while draining the bag
- All working parts are urine free
- No metal parts or sharp corners
- Tube may be cut to desired length



#### FIGURE A - "T"-Bar

An effective corssbar can be made by cutting 2" (5cm) from the bottom of the drain valve tube and pushing one end through the hole in the lever until it is centered, forming a "T" shaped bar.





MADE IN

JSA



LEARN MORE

Sold by the each, 1 valve per package. These are typically stocked items. Non-Sterile, DEHP & Latex-Free.

| Item No.<br>700172 | Description<br>Extra-Large | Size <sup>†</sup><br>0.5" x 72" (183cm) Tube Length |
|--------------------|----------------------------|---|
| 700118             | Large                      | 0.5" x 18" (45cm) Tube Length                       |
| 700110             | Standard                   | 0.5" x 10" (25cm) Tube Length                       |

*†* Approximate dimensions in inches(in) and centimeters(cm).

Case Item No. Not Available Not Available 7011012 (12/CS) 700110200 (200/CS) IIII 10" (25cm) ~ 72" (183cm) III

## Adaptors and Drain Valves

### Urocare<sup>®</sup> Replacement Adaptors, Valves and Gasket-Rings

The following contain all replacement adaptors, valves and gasket-rings which are included in Urocare<sup>®</sup> products that are sold separately. If you are unable to find an item, chances are the item is listed under another category or it is not sold separately.

### Urocare<sup>®</sup> Male Urinal to Latex Leg Bag Adaptor

This replacement adaptor allows for a direct connection between Urocare® Reusable Latex Leg Bags and Urocare® Male Urinal Sheaths. Only for use with Urocare<sup>®</sup> products.

**ATEX-FREE** 

Sold by the each, 1 adaptor per package. This is a stocked item. Non-Sterile, Latex-Free

Dimensions





Item No. Description 6022 Standard Top Adaptor 1.24" x 0.84" (3.15cm x 2.13cm)



### Urocare<sup>®</sup> Straight-Thru Adaptor with Thumb Clamp


This short, multi-purpose adaptor may be used as a replacement adaptor for the Standard Bottom Drain Valve (\*6004) supplied with all Urocare<sup>®</sup> Reusable Latex Urinary Leg Bags and Urocare<sup>®</sup> Male Urinal Sheaths.

It may serve as an alternative drain valve, an extension between Urocare<sup>®</sup> Male Urinal Sheaths and Urocare<sup>®</sup> reusable latex urinary leg bags or Uro-Safe<sup>®</sup> disposable vinyl urinary leg bags or as an adaptor for Urocare<sup>®</sup> Male Urinal Sheaths to connect them directly to most any reusable or disposable urinary leg bag, urinary drainage bag or other urinary and ostomy appliances which accept tubing as a method for connection. Includes small, black Gasket-Ring (#5999) and Two-Position Thumb Clamp (<sup>#</sup>7000) preassembled.

Sold by the each, 1 adaptor per package.

This is a stocked item. Non-Sterile, DEHP & Latex-Free.

Item No. Description 6005 Straight-Thru Adaptor with Clamp

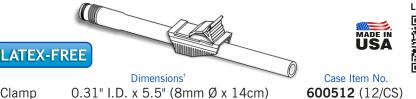





600512 (12/CS)

### Urocare<sup>®</sup> Standard Bottom Drain Valve

Features sealed threads so fingers don't come in contact with urine while draining. Only requires 1<sup>1</sup>/<sub>2</sub> turns to open or close. Replacement drain valve for use with all Urocare<sup>®</sup> Reusable Latex Leg Bags and Urocare<sup>®</sup> Male Urinal Sheaths. Includes Small Black Gasket-Ring (#5999) preassembled.

Sold by the each, 1 valve per package. This is a stocked item. Non-Sterile, Latex-Free,

Item No. Description 6004 Standard Bottom Drain Valve **ATEX-FREE** 

1.53" x 0.6" (3.88cm x 1.52cm)

**Dimensions**<sup>†</sup>





Case Item No. 600410 (100/CS)

### Urocare<sup>®</sup> Urinary Drainage Bottle Adaptor

Provides a secure connection for Urocare<sup>®</sup> Clear-Vinyl Drainage Tubing. Replacement adaptor for use with Urocare<sup>®</sup> Urinary Drainage Bottle. May also be used with Urocare<sup>®</sup> Reusable Latex Leg Bags as a replacement for the Standard Top Adaptor (\*6002) for those who use clear-vinyl tubing.

Sold by the each, 1 adaptor per package. This is a stocked item. Non-Sterile, Latex-Free,

Item No. Description 6003 Drainage Bottle Adaptor ATEX-FREE **Dimensions**<sup>†</sup>

1.24" x 0.84" (3.15cm x 2.13cm)

MADE IN JSA

Case Item No. 600310 (10/CS)





## Adaptors and Drain Valves

### Urocare<sup>®</sup> Replacement Adaptors, Valves and Gasket-Rings

### Urocare<sup>®</sup> Standard Top Adaptor

Features a graduated top with reverse-grip that provides a secure connection for multiple tubing diameters. Replacement top adaptor for use with all Urocare<sup>®</sup> Reusable Latex Leg Bags.

Sold by the each, 1 adaptor per package. This is a stocked item. Non-Sterile, Latex-Free,

Item No. Description 6002 Standard Top Adaptor **ATEX-FREE** 

Dimensions 1.24" x 0.84" (3.15cm x 2.13cm)



600210 (10/CS)





### Urocare<sup>®</sup> "High-Flow" Little Red Valve™

This new, self-cleaning, patented, anti-reflux valve prevents back-flow while allowing fast, uninterrupted drainage for problemfree use. Don't settle for urine reflux or back-pressure caused by inferior flutter valves. Nothing beats Urocare's new "High-Flow" Little Red Valve<sup>™</sup> for performance, reliability and safety. Replacement anti-reflux valve for use in all Urocare<sup>®</sup> Reusable Latex Leg Bags and Urocare<sup>®</sup> Urinary Drainage Bottle.

ATEX-FREE

Sold by the each, 1 anti-reflux valve per package. This is a stocked item. Non-Sterile, 100% Silicone

Item No. Description 6001 Little Red Valve™

**Dimensions**<sup>†</sup> 0.65" DIA. x 0.77"







Case Item No. 600110 (10/CS)

### Urocare<sup>®</sup> Gasket-Rings, Large

Since Urocare's "High-Flow" Little Red Valve<sup>™</sup> serves not only as an anti-reflux valve, but as a gasket-ring as well, in circumstances where the valve is not used, this gasket-ring must be used in place of the valve to prevent leaks in all Urocare<sup>®</sup> Reusable Latex Leg Bags and Urocare® Urinary Drainage Bottle which incorporate this valve.

Sold by the package, 10 rings per package. This is a stocked item. Non-Sterile, Contains Latex

Item No. Description **Dimensions**<sup>†</sup> Gasket-Rings, Large 6000 3" x 5" x 0.25"




### Urocare<sup>®</sup> Gasket-Rings, Small

Replacement gasket-ring for: Urocare<sup>®</sup> Standard Bottom Drain Valve (\*6004), Straight-Thru Adaptor (#6005), Male Urinal Adaptor (#6022), and all Quick Drain Valves™ (#700110, #700118 & #700172).

Sold by the package, 10 rings per package. This is a stocked item. Non-Sterile, Contains Latex

Item No. Description 5999 Gasket-Rings, Small

**Dimensions**<sup>†</sup> 3" x 5" x 0.25"

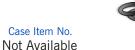



## HCPCS Code Reference List

| ITEM # CODE  | ITEM # CODE   | ITEM # CODE   | ITEM # CODE |
|--------------|---------------|---------------|-------------|
| 99544 A4329  | 6355 A5114    | 520135 A4349  | 4101        |
| 99532        | 6351 A5114    | 520130 A4349  | 4100        |
|              |               |               | 4100 A3102  |
| 99526 A4329  | 6350 A5114    | 520125 A4349  |             |
| 99516 A4329  | 6349A5114     | 5200          |             |
| 99510 A4329  | 6348 A5114    | 5124535 A4349 |             |
| 98532 A4329  | 6347 A5114    | 5124530 A4349 |             |
|              |               |               |             |
| 98526 A4329  | 6346 A5114    | 512435 A4349  |             |
| 98518 A4329  | 6345 A5114    | 512430 A4349  |             |
| 98516 A4329  | 6344 A5114    | 511435 A4349  |             |
| 98510 A4329  | 6343 A5114    | 511430 A4349  |             |
| 98509        | 6342 A5114    | 511425 A4349  |             |
|              |               |               |             |
| 94404 A5105  | 6313 A4334    | 510435 A4349  |             |
| 94403 A5105  | 6312 A4334    | 510430 A4349  |             |
| 94402 A5105  | 6311 A4334    | 510425 A4349  |             |
| 94401 A5105  | 6310 A4334    | 5122535 A4349 |             |
| 94400        | 6200          | 5122530       |             |
|              |               |               |             |
| 94399 A5105  | 6087 A4331    | 512235 A4349  |             |
| 9544A5112    | 6086 A4331    | 512230 A4349  |             |
| 9532 A5112   | 6085          | 511240 A4349  |             |
| 9526         | 6061          | 511235 A4349  |             |
|              | 6060          |               |             |
| 9516 A5112   |               | 511230 A4349  |             |
| 9510 A5112   | 6022 A9900    | 511225 A4349  |             |
| 8532 A5112   | 6020 A4331    | 510235 A4349  |             |
| 8526A5112    | 6019 A4331    | 510230 A4349  |             |
| 8518         | 6018 A4331    | 510225 A4349  |             |
| 8516         | 6017          | 5100          |             |
| 0110 A5112   |               |               |             |
| 8510 A5112   | 6016 A4331    | 501503 A4455  |             |
| 8509 A5112   | 6014 A4331    | 501003 A4455  |             |
| 77321 A4358  | 6013 A4331    | 500503 A4364  |             |
| 77320 A4358  | 6010 A4335    | 5005015 A4364 |             |
| 77181        | 6009          | 500340 A9900  |             |
|              |               |               |             |
| 77180 A4358  | 600831 A4331  | 500335 A9900  |             |
| 76321 A4358  | 600828 A4331  | 500330 A9900  |             |
| 76320 A4358  | 600825 A4331  | 500325 A9900  |             |
| 76181 A4358  | 6007 A9900    | 500222 A4335  |             |
| 76180 A4358  | 60062 A5113   | 500220 A4335  |             |
| 7003         | 6006          | 500218        |             |
|              |               |               |             |
| 700216 A5131 | 6005 A4331    | 500003 A4364  |             |
| 700204 A5131 | 6004 A9900    | 5000015 A4364 |             |
| 700172 A4335 | 6003 A9900    | 4613 A4335    |             |
| 700118       | 6002 A9900    | 4610 A9270    |             |
| 700110 A4335 | 6001 A9900    | 4601          |             |
| 700110       |               |               |             |
| 7000         | 6000          | 4600 A9270    |             |
| 6999 A4363   | 5999 A9900    | 4500 A4335    |             |
| 6401 A4334   | 5600 A4455    | 4423 A4359    |             |
| 6400         | 5500          | 4422 A4359    |             |
| 6394         | 5310          | 4421 A4359    |             |
|              |               |               |             |
| 6393 A4359   | 5300 A5126    | 4420          |             |
| 6392 A4359   | 5221540 A4349 | 4410 A4348    |             |
| 6390 A5114   | 5221535 A4349 | 4409          |             |
| 6384A4359    | 5221530 A4349 | 4404          |             |
| 6383 A4359   | 522140 A4349  | 4403          |             |
|              |               | 4403          |             |
| 6382 A4359   | 522135 A4349  |               |             |
| 6380 A5114   | 522130 A4349  | 4401 A4359    |             |
| 6370 A5114   | 521140 A4349  | 4400          |             |
| 6360 A5114   | 521135 A4349  | 4399 A4359    |             |
| 6358         | 521130 A4349  | 4200          |             |
| 6357         | 521125 A4349  | 4140 A5102    |             |
|              |               |               |             |
| 6356 A5114   | 520150 A4349  | 4102 A9900    | 1           |
|              |               |               |             |

Although Urocare has attempted to provide accurate information, Urocare Products, Inc. assumes no responsibility for the accuracy of the information, cannot be held responsible for typographical errors and provides no guarantees that the codes listed above are still active or have not been altered by the Centers for Medicare & Medicaid Services(CMS) that may affect reimbursement. We believe that all codes listed above are accurate to the best of our knowledge at the time of printing and encourage each customer to verify any code before submitted to CMS for reimbursement. All information is provided "as is" without warranty of any kind, either expressed or implied. Urocare shall not be liable for any indirect, special, consequential or incidental damages including, without limitation, lost profits or revenues, loss or damage to data arising out of the use or inability to use codes, damages resulting from use of or reliance on the information presented, even if Urocare has been advised of the possibility of such damages.

39

## Numerical Product Index

| ITEM# PAGE   | I ITEM# PAGE   | I ITEM# PAGE     | I ITEM# PAGE                            |
|--------------|----------------|------------------|---|
|              |                |                  |   |
| 99544        | 6382           | 600910           | 51043550                                |
|              |                |                  |   |
| 99532 1      | 638012 17      | 6009             | 51043523                                |
| 995261       | 6380           | 600831           | 51043050                                |
|              |                |                  |   |
| 995161       | 637012         | 600828           | 510430                                  |
| 995101       | 6370           | 600825           | 51042550                                |
|              |                |                  |   |
| 98532 1      | 636012         | 6007             | 510425                                  |
|              |                |                  |   |
| 98526 1      | 6360           | 6006107&21       | 512253550                               |
| 985181       | 635812         | 60062            | 5122535                                 |
|              |                |                  |   |
| 985161       | 6358           | 6006107&21       | 512253050                               |
|              |                |                  |   |
| 98510        | 635712         | 60067&21         | 5122530                                 |
| 98509 1      | 6357           | 600512 12 & 37   | 51223550                                |
|              |                |                  |   |
| 94404        | 635612         | 6005             | 51223524                                |
| 94403 9      | 6356           | 6004107, 12 & 37 | 51223050                                |
|              |                | 000410           |   |
| 94402 9      | 635512         | 6004 7, 12 & 37  | 512230                                  |
| 04401 0      | 6355           | 6003107&37       |   |
| 94401 9      |                |                  | 51123550 24                             |
| 94400        | 635112         | 60037&37         | 51123524                                |
|              |                |                  |   |
| 94399        | 6351           | 6002107, 12 & 38 | 51123050                                |
| 9544         | 635012 16      | 6002             | 511230                                  |
|              |                |                  |   |
| 9532         | 6350           | 6001106, 12 & 38 | 51122550                                |
|              |                |                  |   |
| 9526 2       | 634912 16      | 60016, 12 & 38   | 51122524                                |
| 95162        | 6349           | 60006&38         | 51023550                                |
|              |                |                  |   |
| 95102        | 634812 16      | 59996, 11 & 38   | 51023524                                |
| 8532         | 634816         | 5600             | 51023050                                |
|              |                |                  |   |
| 8526         | 634712 16      | 5500             | 510230                                  |
| 85182        | 6347           | 5310             | 51022550                                |
|              |                |                  |   |
| 85162        | 634612 16      | 5300             | 510225                                  |
| 85102        | 6346           | 522154050        | 5100                                    |
|              |                |                  |   |
| 8509         | 634512 16      | 5221540          | 501503                                  |
| 7732112      | 634516         | 522153550        | 50100312                                |
|              |                |                  |   |
| 773213       | 634412         | 5221535          | 501003                                  |
| 7732012 3    | 6344           | 522153050        | 50050312                                |
|              |                |                  |   |
| 773203       | 634312         | 5221530          | 500503                                  |
| 7718112      | 6343           | 52214050         | 500501512                               |
|              |                |                  |   |
| 771813       | 634212         | 522140           | 5005015                                 |
| 7718012      | 6342           | 52213550         | 500340                                  |
|              |                |                  |   |
| 771803       | 6313           | 522135           | 500335                                  |
|              |                |                  | 500330                                  |
| 7632112 3    | 6312           | 52213050         |   |
| 763213       | 6311           | 522130           | 500325                                  |
|              |                |                  |   |
| 7632012 3    | 6310           | 52114050         | 500222                                  |
| 763203       | 6200           | 521140           | 500220                                  |
|              |                |                  |   |
| 7618112 3    | 608712         | 52113550         | 50021813                                |
| 76181        | 6087           | 521135           | 50000312                                |
|              |                |                  |   |
| 7618012 3    | 608612         | 52113050         | 500003                                  |
| 761803       | 6086           | 521130           | 500001512                               |
|              |                |                  |   |
| 700324       | 6085           | 52112550         | 5000015                                 |
| 7003         | 6061508 & 34   | 521125           | 461327                                  |
|              |                |                  |   |
| 70021612     | 6061           | 52014050         | 4610                                    |
| 700216       | 6060508 & 34   | 520140           | 4603                                    |
|              |                |                  |   |
| 700204       | 60608&34       | 52013550         | 4600                                    |
|              |                | 520135           | 4423                                    |
| 700172       | 602212 12 & 37 |                  |   |
| 700118       | 6022           | 52013050         | 4422                                    |
|              | 602012         | 520130           | 4421                                    |
| 700110200 36 |                |                  | 4421                                    |
| 70011012     | 6020           | 52012550         | 4420                                    |
|              |                |                  |   |
| 700110       | 601912         | 520125           | 441011                                  |
| 700050       | 6019           | 5200             | 4409                                    |
|              |                |                  |   |
| 700010       | 601812         | 512453550 23     | 4404                                    |
| 7000         | 6018           | 5124535 23       | 4403                                    |
|              |                |                  |   |
| 699950       | 601712         | 512453050 23     | 4402                                    |
| 6999108&35   | 6017           | 5124530 23       | 4401                                    |
|              |                |                  |   |
| 6999         | 601612         | 51243550 23      | 4400                                    |
| 6401         | 6016           | 512435           | 439910                                  |
|              |                |                  |   |
| 6400         | 601410         | 51243050 23      | 4200                                    |
| 6394         | 6014           | 512430           | 41405                                   |
|              |                |                  |   |
| 639315       | 601310         | 51143550 23      | 4102 6                                  |
| 6392         | 6013           | 51143523         | 410112                                  |
| CI           |                |                  |   |
| 639012       | 601050         | 51143050 23      | 4101                                    |
| 6390         | 60101035       | 511430           | 410012 5                                |
|              |                |                  |   |
| 6384 15      | 6010           | 51142550         | 4100                                    |
| 638315       | 600950         | 511425           |   |
|              |                | 511,20           | l i i i i i i i i i i i i i i i i i i i |
|              |                |                  |   |



# 2018 PRODUCT CATALOG

## Providing Quality Health Care Products Since 1975

LEARN MORE

#### MANUFACTURED BY:

Urocare Products, Inc. 2735 Melbourne Avenue Pomona, CA 91767-1931, U.S.A.









Please think of our environment before printing

Within the US and Canada call: 1-800-423-4441,

Requests can also be faxed to 1-909-621-4436 or

Customer Service email: cust.support@urocare.com

outside the US call: 1-909-621-6013

Sales email: sales@urocare.com

visit us on the web at www.urocare.com

FORM NO: CAT2018-81010100B

Publish Date: 05/01/2018

Copyright  $\ensuremath{\mathbb O}$  2018 Urocare Products, Inc. All rights reserved.

**Questions?** Comments?