Urocare Products, Inc. • 2735 Melbourne Avenue • Pomona, California 91767-1931 • U. S. A.
T: (909) 621-6013 • F: (909) 621-4436 • E: sales@urocare.com • I: www.urocare.com

Uro-Bond® III Brush-on Silicone Adhesive

DESCRIPTION

Uro-Bond® III brush-on silicone adhesive is a pressure-sensitive silicone adhesive specifically formulated for unique bonding applications. Supplied as a 50% silicone solids solution in a ethyl acetate solution, it is suitable for adhering non-permanent material to the body surface. Uro-Bond® III's features include retention of adhesive qualities in the presence of moisture or perspiration, adhesive properties are unaffected by normal temperature variations or time and it adheres to many surfaces—skin, metals, glass, paper, fabric, plastics, silicone and organic rubber products.

APPLICATIONS

Uro-Bond® III brush-on silicone adhesive may be used to adhere dressings, prosthetics and metal or plastic devices to the body. Suggested uses include but are not limited to: Securing male external urinary catheters, adhering colostomy, ileostomy and tracheostomy applications, attaching electrodes to the skin for patient monitoring, affixing surgical dressings and pads to the skin and adhering maxillo-facial prosthetic devices. The purchaser should thoroughly test products made in part or otherwise incorporating Uro-Bond® III brush-on silicone adhesive to determine the acceptability of the product's performance in a specific application for his/her own use.

STORAGE AND SHELF LIFE

WARNING: Extremely flammable. Liquid and vapors may ignite. Do not use when smoking. Do not use or store near fire, flame, heat, electric sparks or any other source of ignition. Use only as directed. Keep out of eyes. In case of eye contact, immediately flush eyes with water, remove any contact lenses and continue to flush eyes with plenty of water for at least 15 minutes. Contact physician. Harmful if ingested. Do not inhale or permit adhesive to enter stoma or open wounds. In case of accidental ingestion, give fluids liberally and consult with local POISON CONTROL CENTER. If redness, swelling or rash appears near area of use, discontinue use. Contact Physician. Harmful to synthetic fabrics, wood finishes and some plastics. **Keep out of reach of children.** When stored in its original unopened container at or below 72° F (22° C), Uro-Bond® III has a shelf life of Two (2) years from the date of manufacture. If product thickens during storage, Urocare recommends the use of Uro-Bond® Thinner (#501003) to return Uro-Bond® III to the desired consistency. **PLEASE NOTE: Do not leave Jar open for prolonged periods of time**—close cap securely immediately after application.

INSTRUCTIONS FOR USE

Since Uro-Bond® III is supplied as an 50% silicone solids solution in a ethyl acetate solution, it is normally used as supplied. It is applied with the brush attached to the underside of the cap. To obtain the maximum adhesion capability of Uro-Bond® III, surfaces must be thoroughly cleaned prior to application and allowed to dry thoroughly before contact is made. The adhesive should be applied liberally to both surfaces which are to be adhered; additional coats of adhesive may increase adhesion.

REMOVAL

To remove Uro-Bond® III from the body surface, use Urocare Adhesive Remover Pads (#5600) or another medical tape and adhesive remover. Do not use Uro-Bond® Thinner as an adhesive remover; it is strictly intended for use as a thinning agent only! To remove Uro-Bond® III from surfaces other than the body surface (skin), use one of the following removers or solvents: isopropylalcohol (rubbing alcohol), ethyl alcohol, toluene, xylene, hexane or ethyl acetate; however, it should be noted that these removers or solvents may cause skin irritation and are highly flammable. They may also deteriorate certain surfaces, for example: certain types of plastics, and should be used accordingly. It is the purchaser's responsibility to thoroughly test the use of any remover or solvent to determine suitability of use. Liberally apply the appropriate remover to a cloth or gauze pad and wipe the surface clean.

PACKAGING

Uro-Bond® III brush-on silicone adhesive is supplied in a 3 fl. oz./88.7 ml. (#500003) and 1½ fl. oz./44.4 ml. (#5000015) glass jars with a brush attached to the cap.

ORDERING INFORMATION

Please visit our internet site at: www.urocare.com. Here you will find current product information, as well as the location of your nearest Urocare distributor. If you need assistance, please contact us either via our internet site or call our toll-free number at: (800) 423-4441 Mondays ~ Thursdays from 7:30 AM ~ 5:00 PM (PST).

The information contained herein is based on information we believe to be accurate and reliable; however, Urocare encourages you to thoroughly test any application and independently conclude the effectiveness and suitability of Uro-Bond® III before use. Suggestions for use should not be taken as inducements to infringe upon any patent.

Urocare Products, Inc. • 2735 Melbourne Avenue • Pomona, California 91767-1931 • U. S. A.
T: (909) 621-6013 • F: (909) 621-4436 • E: sales@urocare.com • I: www.urocare.com

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