



# TRACHO-FOAM<sup>®</sup> DISCS

For Securing Valve Housings of Voice  
Restoration Systems to the Neck



Product #5300 Shown

## FEATURES INCLUDE:

- Comfortable air-tight seal
- Conforms easily to the irregular contours of the body
- Moisture resistant & eliminates hissing

ITEM #	DESCRIPTION & SIZE
5320	Extra-Large, 1/8" x 1" I.D. x 3 1/2" O.D.
5310	Large, 1/8" x 1" I.D. x 2 5/8" O.D.
5300	Standard, 1/8" x 1" I.D. x 2 1/4" O.D.


## FOR BEST RESULTS:

Always apply Tracho-Foam<sup>®</sup> discs to a thoroughly cleaned and dry area of skin larger than the disc itself and centrally located to the stoma.

All residue, adhesives, etc. must be removed from the flange of the voice restoration system and the flange thoroughly cleaned before application.

## APPLICATION INSTRUCTIONS:

1. Peel paper backing away from **one side only** of the disc—see Figure 1. Be careful not to touch the adhesive area of the disc.
2. With the adhesive side of the disc facing the valve housing flange, adhere the disc centrally to the flange. Using your fingers, press gently against the disc to remove any air pockets that may be trapped between the disc and the flange.
3. Peel away second paper backing as shown in Figure 2.
4. Very carefully set the voice restoration system with the adhesive disc attached to the flange to the neck. For best results, be certain to set the unit as close to the center as possible.
5. Gently press the unit with the disc attached against the neck to remove any air that may be trapped between the disc and the skin to ensure a reliable, comfortable seal.

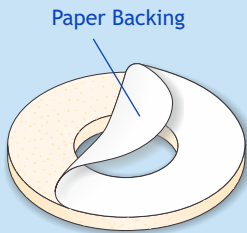

**FIGURE 1**

## DISC REMOVAL:

To remove Tracho-Foam<sup>®</sup> discs from skin or appliances, gently begin peeling disc from edge and continue until entire disc has been removed.

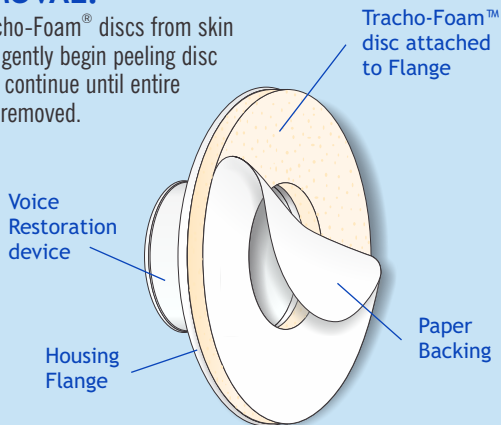


FIGURE 2

## Having trouble removing Tracho-Foam<sup>®</sup> discs?

Urocare<sup>®</sup> Adhesive Remover Pads may be used to soften the adhesive and assist in the removal of the disc. They contain a citrus-based, non-toxic adhesive remover that may also be used to remove most types of medical adhesive or adhesive residue from skin or appliances or in preparation for use of a new disc.

Most medical grade adhesive removers will work basically in the same manner; however, consult your doctor or speech therapist to decide the best product for you and how to use it safely.



Product #5600 Shown



Quality Health Care  
Products Since 1975



## RELATED PRODUCTS

### Uro-Bond<sup>®</sup> III & IV

Brush-on Silicone Adhesives & Thinner

### Urofoam<sup>®</sup> 1 & 2

Single & Double-Sided Adhesive Foam Strips

### Urocare<sup>®</sup> Adhesive Remover Pads

for dissolving and removing medical adhesives

### Uro-Prep<sup>®</sup> Protective Skin Barrier Wipes

forms a protective barrier between aggressive adhesives and delicate skin

For more information on any of Urocare's other products, contact your nearest Urocare dealer or write to:

Urocare Products, Inc.  
Customer Service Department  
2735 Melbourne Avenue  
Pomona, CA 91767-1931, U.S.A.  
T: (909) 621-6013 • F: (909) 621-4436  
E: [sales@urocare.com](mailto:sales@urocare.com) • I: [www.urocare.com](http://www.urocare.com)



PLEASE RECYCLE  
Printed on  
Recycled Paper

FORM NO: PI5300-79228000C