



# CATHETER/DRAINAGE TUBE STRAP

For use with most foley catheters and urinary drainage tubes

- Holds foley catheters and drainage tubing securely in position for optimum flow
- Minimizes catheter and drainage tube movement trauma
- Fully adjustable self-gripping fasteners make application safe and simple
- Durable, washable and reusable

LEARN MORE



REF/ITEM #	DESCRIPTION & SIZES <sup>†</sup>
<b>6310</b>	Standard 18" (46cm) Fits: 17" ~ 58" DIA. (43cm ~ 147cm Ø)
<b>6311</b>	Arm 14" (36cm) Fits: 11" ~ 37" DIA. (28cm ~ 94cm Ø)
<b>6312</b>	Large 22" (56cm) Fits: 7" ~ 24" DIA. (18cm ~ 60cm Ø)
<b>6313</b>	Abdominal 34" (86cm) Fits: 9" ~ 30" DIA. (23cm ~ 78cm Ø)

<sup>†</sup> Approximate dimensions in inches(") and centimeters(cm).

For more information on this or any other Urocare product, please visit our website at [www.urocare.com](http://www.urocare.com). To place an order, please contact your health care provider.

Manufactured and distributed by:

Urocare Products, Inc.  
2735 Melbourne Avenue  
Pomona, CA 91767-1931  
U.S.A.



US and Canada: +1 (800) 423-4441  
Outside the US: +1 (909) 621-6013



[sales@urocare.com](mailto:sales@urocare.com)



PLEASE RECYCLE

FORM NO: PI6310EN-10802000

## READ THIS FIRST:

For best results, take certain precautions before using these straps. Be certain that there is no loose stitching or other irregularities before each use.

Those with known allergic reactions to certain materials (especially neoprene rubber) should not wear these straps. Discontinue use at first signs of dermatitis or any other type of skin irritation and contact a physician immediately.

**Do not over-tighten strap:** Products made with elastic and/or neoprene rubber material can—if improperly fit—cause circulation restriction or skin breakdown; however, a catheter tube strapped too loosely can slip down the leg and cause trauma by stretching it. Should any discomfort occur while wearing, discontinue use immediately and contact a fitter or physician.

To avoid possible drainage problems, read all instructions and precautions carefully before use.

## APPLICATION INSTRUCTIONS:

1. Position the strap and catheter or tube around the limb and stretch into place to fit snugly as shown in figure 1.
2. Wrap the strap tie over, around and behind the catheter or tube and insert through hole in strap tie as shown in figure 2.
3. Pull strap tie until it tightens around the catheter or drainage tube.
4. Then fold strap tie over and secure into place as shown in figure 3.

**IMPORTANT:** The number of times the hook & loop fasteners are opened and closed determines the life of the fasteners. Do not open and close the fasteners unnecessarily—this will ensure a long life.

## LAUNDERING INSTRUCTIONS:

Join ends of strap and close strap tie before washing. Hand or machine wash cold—gentle/delicate cycle. Do not bleach, do not tumble dry, do not wring, dry in shade, do not iron, do not dry clean.

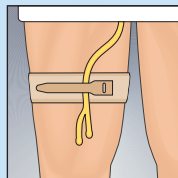
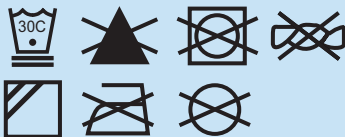


FIGURE 1

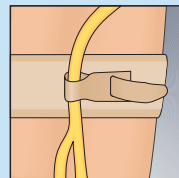


FIGURE 2

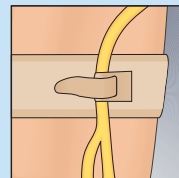


FIGURE 3