



UROCARE®

URINARY DRAINAGE BAG



Sterile only if package is undamaged and unopened. Avoid freezing or excessive heat. Store in a cool, dark, dry place. Do not resterilize.

INSTRUCTIONS FOR USE

Wash Hands: Wash your hands with soap and water thoroughly before handling any urinary supplies.


Connecting the Drainage Bag: Ensure the drainage bag tubing and catheter end are clean and dry. Insert the nozzle adapter at the end of the tubing on the bag securely into the end of the catheter; twist gently if needed—do not force.

Positioning the Bag Properly: Always keep the drainage bag below the level of the bladder to prevent back-flow, which can cause infections (UTI). Use the bed rail hooks to secure the bag to the side of the bed or patient's wheelchair. Make sure the tubing is not kinked or clamped and allows urine to flow freely.

Emptying the Bag: Empty when the bag is about ¾ full or at least every 24 hours. To empty:

1. Hold the bag over a toilet or container.
2. Open the T-Tap drain valve at the bottom.
3. Avoid letting the outlet touch the toilet or container.
4. Close T-Tap drain valve and wipe the valve with alcohol or a disinfectant wipe.
5. Wash hands thoroughly again.

Scan here for more product information

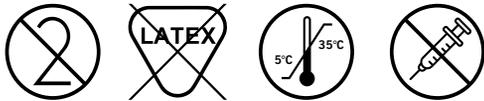

UDI




(01)00792293042002
(11)20250830
(17)20300829
(10)255171
(30)1

Urocare Products, Inc.
2735 Melbourne Avenue
Pomona, CA 91767, U.S.A.

Questions? Comments? Call us
US and Canada: +1 (800) 423-4441
Outside the US: +1 (909) 621-6013

sales@urocare.com

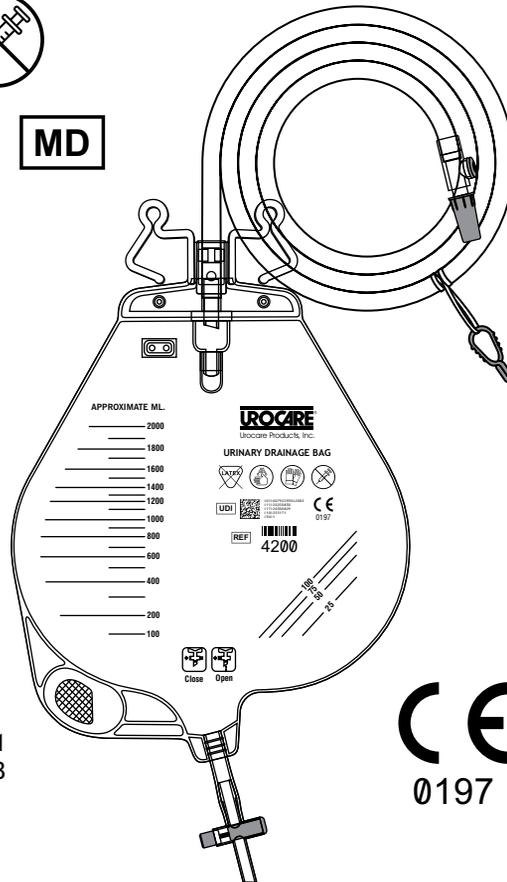

REF 4200

QTY 1 U/M EA

LOT 255171

REF 20250830

REF 20300829

CE 0197

MADE IN CHINA

UPC-A 7 92293 04200 2



FORM NO: SPW4200L-997210001